



Instruction Sheet

Therapy Net 0130

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises

MATERIALS LIST:

- 1 - Net with Welded Steel Rings Attached
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED:

Minimum height required is 8 ft.
Caution must be used to keep a clear distance of 6 ft. from walls and other obstacles.

HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED:

The use of a Southpaw Height Adjuster (# 5000) is recommended for ease of adjusting your equipment to the height of your client.

The Therapy Net, as with all Southpaw suspended equipment, **requires** the use of a Southpaw Safety Rotational Device (# 2000) to prevent injury to your clients and damage to your equipment.


WORKING LOAD: 200 lbs.

HANGING PROCEDURE:

Your Therapy Net can be suspended from one or two points using your Southpaw Safety Snaps (# 0100).

Your Therapy Net can be combined with other Southpaw equipment to create a number of beneficial exercises.

It can be hung from a Bolster Bar, used with a Southpaw Floor Bubble, or a Southpaw Net Adaption Kit in the net.

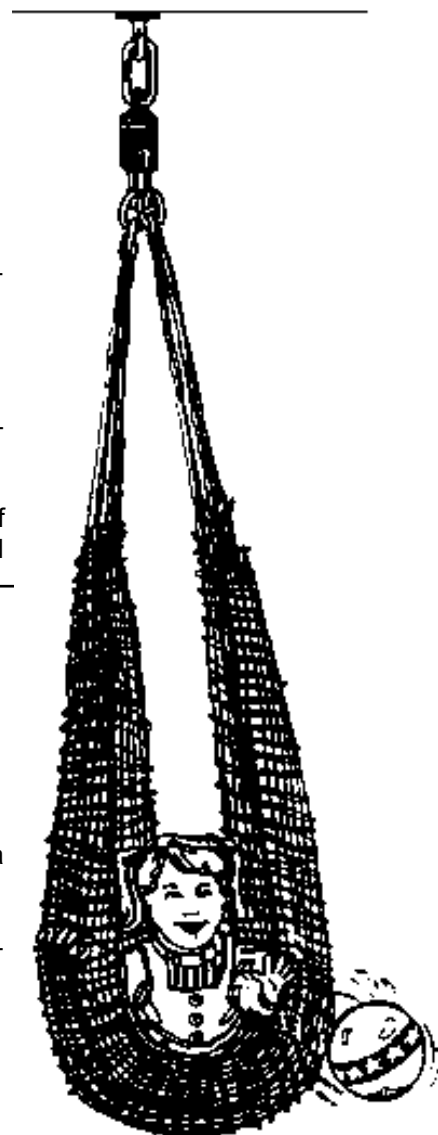

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for wear at the top where the welded rings contact the net.
- Check for wear where the rings contact the safety snaps.
- Check for wear where client sits.

MATTING PRECAUTIONS: Always use appropriate matting.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by client without proper supervision.

CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- May be hand washed with mild soap or detergent.
- Hang to dry before using.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Please visit us at www.southpaw.com

© 2019 Southpaw Enterprises, Inc.