



Instruction Sheet

Net Adaptation Kit 0131

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- | | |
|---------------------------|---------------------------|
| 1 - PADDED SEAT | 1 - INSTRUCTION SHEET |
| 1 - TOP SPREADER BAR WITH | 1 - MAINTENANCE CHECKLIST |
| 2 - RETAINING PINS | |

Warning: Failure to properly install a forged steel eye bolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

IMPORTANT: KEEP ALL PARTS TOGETHER

If any of the above materials are missing, please contact Southpaw Enterprises immediately.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Same as for the Southpaw Therapy Net: 8 feet
- Use caution to keep a clear distance of 6 feet from all walls and other obstacles

HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

- A Southpaw Height Adjuster is recommended for ease in adjusting your net to the correct height.
- A Southpaw Safety Rotational Device must be used to prevent stress on your suspension system and damage to your net

WORKING LOAD

- 200 lbs. when used with Southpaw Therapy Net (Not included).

Typical Seating Styles

1. Remove the two retaining pins from the top spreader bar and set aside. Do not lose these pins. Slip the upper cords at the top of a Southpaw Therapy Net into the slots at either end of the top spreader bar (see Fig. 1), and replace the retaining pins. The insert seat is simply positioned in the bottom of the net. It can be used with the client sitting with the net to either side (see Fig. 1) or straddling the net in front, (see Fig. 2).
2. For extra back support: begin by following step one. With the spreader bar and seat in place, gather two or three loops of a section of the Therapy Net on one side. Slide these loops into the slot of the spreader bar on the same side (fig 3a). Replace the

retaining pin into that side of the spreader bar, making sure all of the

Fig. 1

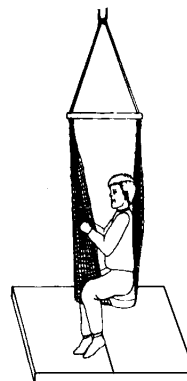


Fig. 2






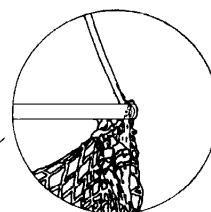


Fig. 3a

Fig. 3

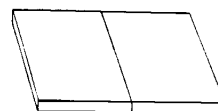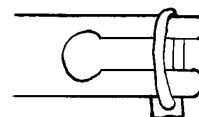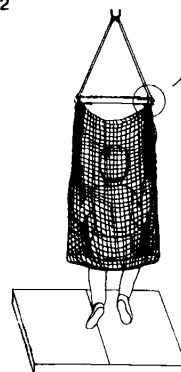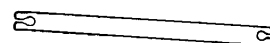

Seat

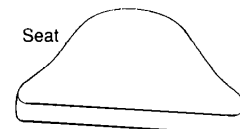

Spreader Bar

therapy Net is "trapped" by the pin. Repeat this step for the other side of the Therapy Net(Fig 3). Adjust as needed.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check the top spreader bar for cracks or splinters. If cracks are found, discontinue use. If splinters are found, sand smooth and refinish.
- Check seat insert for worn upholstery or broken / damaged zipper, if damaged discontinue use immediately until repaired.
- Always check height adjustment system before using to see that it is working properly.
- Check net wear points before using with the Net Adaptation Kit.

STORAGE PROCEDURES

- Store out of direct sunlight
- Store where not accessible by clients without proper supervision

CLEANING INSTRUCTIONS

- Each piece of the Net Adaptation Kit can be cleaned with a damp cloth and mild disinfectant

MATTING PRECAUTIONS:

- FLOOR MATS OF PROPER THICKNESS SHOULD BE POSITIONED APPROPRIATELY UNDER AND AROUND THE SUSPENDED EQUIPMENT.

CAUTION

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at southpaw.com

© 2024 Southpaw Enterprises, Inc.