

Instruction Sheet Child Cocoon Swing 0132

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Warning: Failure to properly install a forged

steel eyebolt (1/2" or 5/8") may result in serious

injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises

Customer Service at 1-800-228-1698.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

COCOON SWING

- 1 Cocoon Swing
- 2 Safety Snaps
- 1 Safety Snap Instruction Sheet
- 1 Maintenance Checklist
- 1 Instruction Sheet

WORKING LOAD:

Child Cocoon Swing working load is 150 pounds.

HEIGHT REQUIREMENTS:

 The Cocoon Swing is approximately 60" long and can be used with 8' ceilings. Use caution to keep a minimum of 6' clear distance from walls and other obstacles. Client may need assistance to enter the Cocoon Swings.


HEIGHT ADJUSTER AND SAFETY ROTATIONAL REQUIREMENTS:

 To hang your Southpaw Cocoon Swing you may need to use some of the Southpaw Height Adjustment Equipment such as Therapy Rope (#0150), Eye Splice (#0151), and Height Adjuster (#5000). A Southpaw Safety Rotational Device (#2000) must be used to prevent damage to hanging equipment.

HANGING AND USE

Your Cocoon Swing comes with Safety Snaps attached to the loops on the ends of the web. Simply attach the Snaps to your suspension system. Adjust to the appropriate height for your client.

SPECIAL NOTES: The Child Cocoon Swing can be used with all straps connected or with the front and center back disconnected to form a single point Sling Hammock. The front entry door can be lowered to allow for easy access getting in or out of the Cocoon Swing. See the drawing at right.

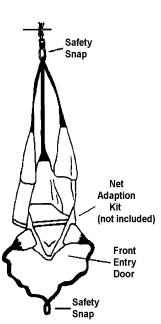

CLEANING AND STORAGE PROCEDURES:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Wipe vinyl clean with a damp cloth using soap and water or disinfectant.
- Do not immerse in water.
- Knit jersey may be hand washed using care not to immerse vinyl.
- Store out of direct sunlight.
- Store where not accessible to clients without proper supervision.

We do not recommend the use of aluminum safety snaps or caribiners although they may be lighter in weight or less expensive. Aluminum snaps will wear much faster than steel snaps. They may also present a health hazard from handling and ingestion of aluminum particles from that wear.

MAINTENANCE CHECKLIST:

- All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.
- Check Safety Snaps for wear, see enclosed Safety Snap Instruction Sheet for wear points and maintenance.
- Check the web loops at the point that they rub against the Safety Snaps for wear. If excessive wear is found, discontinue use, and contact Southpaw for repairs.
- Check all seams for loose and missing stitches. If loose or missing stitches are found in the seams joining two pieces of fabric together, fabric to vinyl, or web to vinyl, discontinue use until repaired.

CAUTION -

- Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.
- ♦ Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at southpaw.com
© 2023 Southpaw Enterprises, Inc.