



Instruction Sheet

Tube Wrap Single 0185

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a Southpaw forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the Southpaw forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST:

TUBE WRAP SINGLE (0185)
1 - Tube Wrap
1 - Valve Stem Sleeve
1 - Instruction Sheet
1 - Maintenance Checklist

TUBE WRAP SET OF TWO (0186)
2 - Tube Wraps
2 - Valve Stem Sleeves
1 - Instruction Sheet
1 - Maintenance Checklist

ASSEMBLY PROCEDURES:

Inflate tube to desired size.

Place the hole in the Tube Wrap over the valve stem with the valve stem pointing toward the short end of the wrap.

Slip the valve stem sleeve over the valve stem.

Wrap the Tube Wrap around the tube and press the hook and loop fastener together securely. Check to make sure the valve stem is under the foam padding and between the two strips of hook and loop fastener. If not, readjust the wrap before using.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

Check Tube Wrap for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.

Check hook fastener for lint or debris. If lint is found in the hook fasteners, it may be removed with a brush or pick.

MATTING PRECAUTIONS:

Always use appropriate matting.

As with all suspended equipment we suggest the client wear a helmet.

STORAGE PROCEDURES:

Store out of direct sunlight.

Store away from all chemicals and household cleaners.

Store where not accessible by clients without proper supervision.

CAUTION: Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.

Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at southpaw.com

© 2024 Southpaw Enterprises, Inc.



EQUIPMENT MAINTENANCE CHECKLIST

Inspection Action: (I) Inspect (R) Replace

[illegible]

* This sheet is for example only. It is not meant to, in any way, replace county, state, or federal standards.