



SOUTHPAW®

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet

Bolster Swing
1000

Deluxe Bolster Swing
1001

Pony Bolster Swing
1005

Deluxe Pony Bolster Swing
1015



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

MATERIALS LIST

STANDARD AND PONY BOLSTER

- 1 - Bolster with built in Safety Rotational Device and 6 Safety Snaps
- 1 - Instruction sheet for Bolsters and for Safety Snaps
- 1 - Maintenance Checklist

DELUXE BOLSTER AND DELUXE PONY BOLSTER

- 1 - Bolster with built in Safety Rotational Device and 6 Safety Snaps
- 1 - Carpet
- 1 - Instruction sheet for Bolsters and for Safety Snaps
- 1 - Maintenance Checklist

IMPORTANT: KEEP ALL PARTS TOGETHER

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Can be used with an 8 foot ceiling
- Caution must be taken to keep a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER NEEDED

- For ease of adjustment, a Southpaw Height Adjuster (Part # 5000) is recommended

WORKING LOAD

- 350 POUNDS

ASSEMBLY PROCEDURES

(No assembly required)


HANGING PROCEDURE

To hang your bolster attach the safety snap on the top of the bolster to your suspension point. Your Southpaw Bolster is now ready to use.

CONFIGURATION OF UPPER WORKS

For variations during therapy, the upper works may be connected to the Bolster Swing in different ways. You may keep all three ropes connected, detach one end or the other (shown), detach both ends, or detach the middle.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

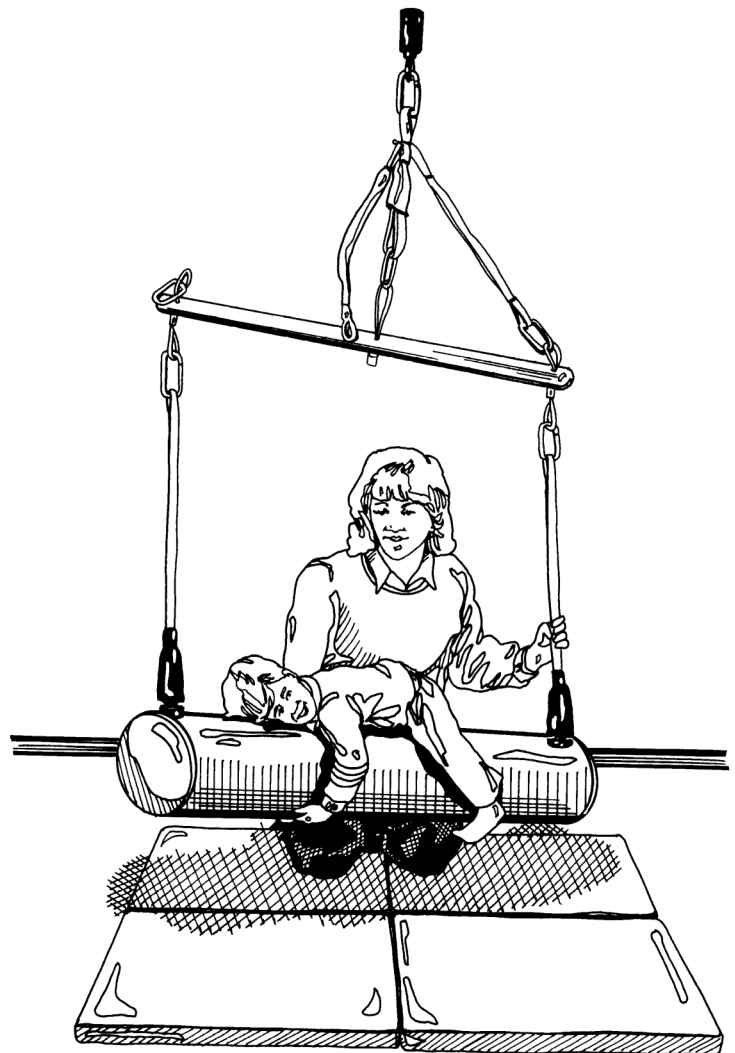

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed

- Check safety rotational device for wear on the safety snap and eyebolt. If found on eyebolt, discontinue use until repaired. If found on safety snap, discontinue use until replaced.
- If device does not spin smoothly or makes noise discontinue use.
- Check ropes for fraying (be sure to check end of rope under black boot).
- Check loops for fraying or split threads discontinue use if found.
- Check all Safety Snaps for wear. Refer to the Safety Snap Instruction Sheet for detailed maintenance checklist.
- Check ropes for loose or missing metal rope clips. If found discontinue use.
- Check bar for cracks or splinters and loose bolts. If cracks are found discontinue use. If splinters are found sand until smooth. Tighten loose bolts. Apply Loctite 242 to threads if persistently loose.

STORAGE PROCEDURES

- Store out of Direct Sunlight
- Store where not accessible by clients without proper supervision

CLEANING INSTRUCTIONS

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Wash using a damp cloth with mild soap and water.
- Do not use abrasive cleaners.

CAUTION

- ☠ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ☠ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at www.southpaw.com

© 2019 Southpaw Enterprises, Inc.