



Store these instructions in a safe place or with the enclosed maintenance checklist

Instruction Sheet

Standard Bilateral Bolster 1070

Pony Bilateral bolster 1075



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

Standard or Pony Bilateral Bolster

- 1 - Bolster with built in Safety Rotational Device and 6 Safety Snaps
- 1 - Instruction Sheet
- 1 - Safety Snap Instruction Sheet
- 1 - Maintenance Checklist

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

IMPORTANT: KEEP ALL PARTS TOGETHER

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Can be used with an 8 foot ceiling
- Caution must be taken to keep a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER NEEDED

- For ease of adjustment, a Southpaw Height Adjuster (Part # 5000) is recommended.

WORKING LOAD

- 350 lbs.

ASSEMBLY PROCEDURES

No assembly required.


HANGING PROCEDURE

To hang your bolster attach the safety snap on the top of the bolster to your suspension point.

Your Southpaw Bilateral Bolster is now ready to use.

CONFIGURATION OF UPPER WORKS

For variations during therapy, the upper works may be connected to the Bilateral Bolster Swing in different ways. You may keep all three ropes connected, detach one end or the other (shown), detach both ends or detach the middle.

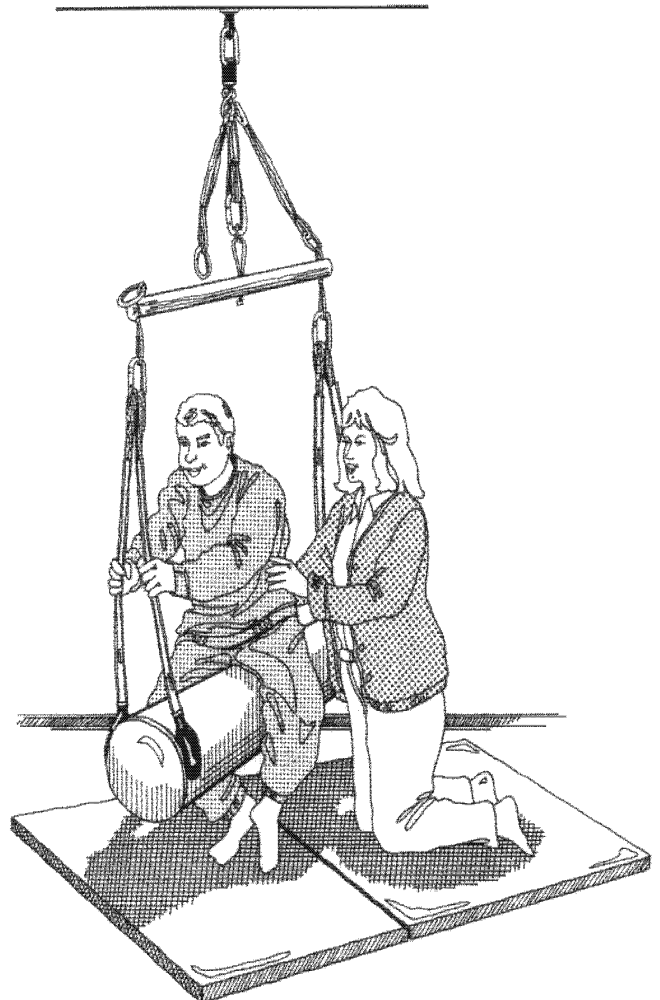

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check safety rotational device for wear on the safety snap and eyebolt. If found on eyebolt, discontinue use until repaired. If found on safety snap, discontinue use until replaced.
- If device does not spin smoothly or makes noise discontinue use.
- Check ropes for fraying (be sure to check end of rope under black boot).
- Check wire loops for broken or split wires discontinue use if found.
- Check all Safety Snaps for wear. Refer to the Safety Snap Instruction Sheet for detailed maintenance checklist.
- Check ropes for loose or missing metal rope clips. If found discontinue use.
- Check bar for cracks or splinters. If cracks are found discontinue use. If splinters are found sand until smooth.

STORAGE PROCEDURES

- Store out of Direct Sunlight
- Store where not accessible by clients without proper supervision

CLEANING INSTRUCTIONS

- Wash using a damp cloth with mild soap and water
- Do not use abrasive cleaners

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at www.southpaw.com

© 2019 Southpaw Enterprises, Inc.