



# SOUTHPAW®

Store these instructions in a safe place or with the enclosed maintenance checklist

## Instruction Sheet

### Standard Bilateral Bolster - Body Only 1071

### Pony Bilateral bolster - Body Only 1076



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

#### MATERIALS LIST

##### Standard or Pony Bilateral Bolster

- 1 - Bolster with Side Lines
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

**IMPORTANT: KEEP ALL PARTS TOGETHER**

#### MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Can be used with an 8 foot ceiling
- Caution must be taken to keep a clear distance of 6 feet from walls and other obstacles.

#### WORKING LOAD


- 350 POUNDS

#### ASSEMBLY PROCEDURES

Remove your Southpaw Standard bolster from the upper works by detaching the side lines from the Safety Snaps. Leave the Safety Snaps attached to the wire loops. Hang your Bilateral Bolster by attaching the two side lines on one end of the bolster to the Safety Snap on one end of the upper works. (Be sure that the seam in the cover is on the bottom). Attach the two side lines on the other end of the Bolster to the Safety Snap on the other end of the upper works. Your Southpaw Bilateral Bolster is now ready to use.

#### CONFIGURATION OF UPPER WORKS

For variations during therapy, the upper works may be connected to the Bilateral Bolster Swing in different ways. You may keep all three ropes connected, detach one end or the other (shown), detach both ends, or detach the middle.

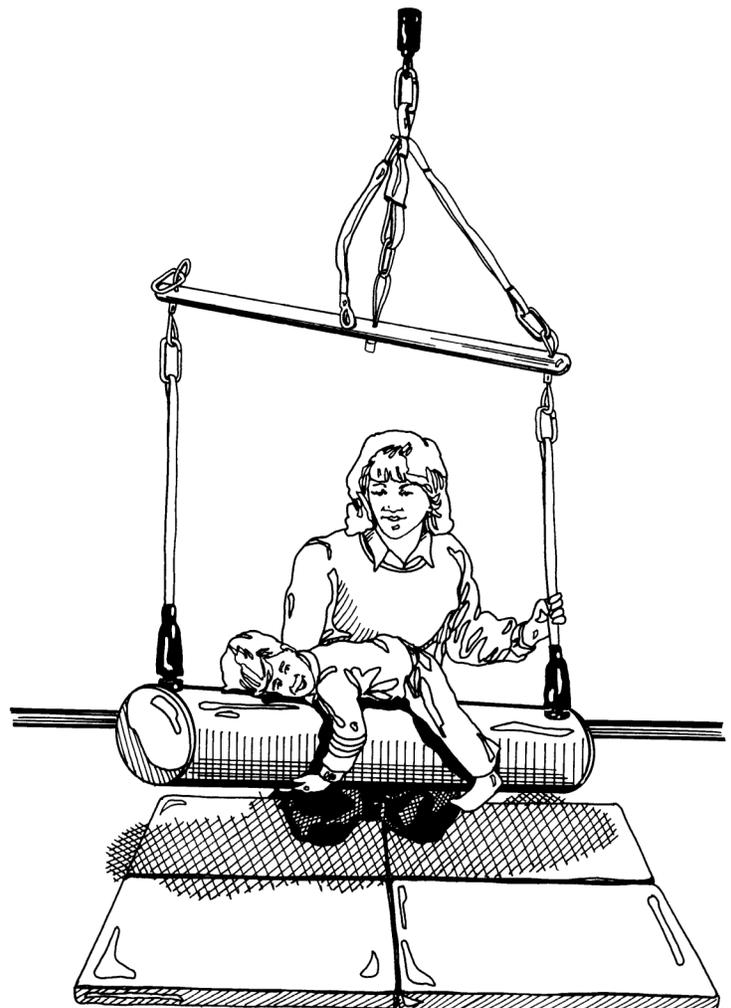

## MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check safety rotational device for wear on the safety snap and eyebolt. If found on eyebolt, discontinue use until repaired. If found on safety snap, discontinue use until replaced.
- If device does not spin smoothly or makes noise discontinue use.
- Check ropes for fraying (be sure to check end of rope under black boot).
- Check wire loops for broken or split wires discontinue use if found.
- Check all Safety Snaps for wear. Refer to the Safety Snap Instruction Sheet for detailed maintenance checklist.
- Check ropes for loose or missing metal rope clips. If found discontinue use.
- Check bar for cracks or splinters. If cracks are found discontinue use. If splinters are found sand until smooth.

## STORAGE PROCEDURES

- Store out of Direct Sunlight
- Store where not accessible by clients without proper supervision

## CLEANING INSTRUCTIONS

- Wash using a damp cloth with mild soap and water
- Do not use abrasive cleaners

## CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at [www.southpaw.com](http://www.southpaw.com)

© 2019 Southpaw Enterprises, Inc.