



Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet

Additional Brackets-Home Therapy System 116011



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.


MATERIALS LIST:

- 2 - Brackets
- 6 - # 12 x 1-1/2" Screws
- 1 - 3/16" Drill Bit
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

INSTALLATION:

To install your brackets you will need a drill, a 3/16" drill bit (furnished), and a phillips head screwdriver.

- Position the bracket on the door frame against the door stop molding, on the opposite side of the molding from the door. See figure 1. Put the side of the bracket with the three small holes against the door frame. On a standard 7 ft. door position the top of the bracket against the top of the door frame.
- Using the 3/16" drill bit provided, drill a pilot hole into the door frame using the small hole in the bracket as a guide. Put a screw through the bracket into the drilled hole and tighten the screw securely. This will hold the bracket in place while you drill the two remaining holes and install the screws. Make sure all screws are tight, but be careful not to over tighten enough to strip the screw threads.
- Install the other bracket on the opposite side of the door frame in the same manner.

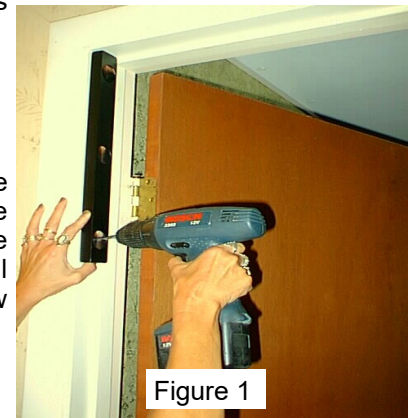

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at www.southpaw.com

© 2020 Southpaw Enterprises, Inc.