



southpaw enterprises, inc.®

Instruction Sheet

Southpaw Disc Swing 1172

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

- 1 - Disc Swing
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

Allow a 6 ft. clearance in all directions.

WORKING LOAD: 150 lbs.

A Safety Rotational Device must be used at all times. All warranties are void if a Southpaw 2000 Rotational is not used or used with a rotational device not approved by Southpaw Enterprises, Inc.

Note: A Southpaw 5000 Height Adjuster may be needed to adjust your equipment to the correct height for your client.

HANGING INSTRUCTIONS

- Attach the loop at the end of the rope to your Safety Rotational Device. Adjust height as necessary.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check all metal to metal connections for wear. Discontinue use if the loop shows excessive wear.
- Check the Safety Rotational Device. (See separate Instruction Sheet).
- Check the ropes for loose or missing splice locks. The splice lock is the metal staple in the rope. One splice lock is located at the top and bottom of the swing.
- Check the rope for wear. When the rope shows excessive wear, contact Southpaw to replace the rope.
- Check the plastic flange on the bottom of the Disc Swing seat to insure that it is not cracked or broken. Discontinue use if the plastic flange is cracked or broken.
- Check ceiling eye bolt for wear and movement. If ceiling eye bolt moves more than 1/4", discontinue use of suspension point until repaired.

MATTING PRECAUTIONS

- Always use appropriate matting

STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean plastic part of the Disc Swing with a damp cloth using soap and water or disinfectant.
- Dry before using.
- Do not immerse in water.

Note: When replacing Safety Snaps or Rotational Devices due to normal wear, use only Southpaw replacement parts.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>