



# Instruction Sheet

## Moon Ball Swing 120175

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST:

- |                                 |                                   |
|---------------------------------|-----------------------------------|
| 1 - Moon Ball Swing Cover       | 1 - Instruction Sheet             |
| 1 - Therapy Ball                | 1 - Safety Snap Instruction Sheet |
| 4 - Locking Safety Snap         | 1 - Maintenance Checklist         |
| 6 - Single Loop Replacement VSD |                                   |

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED:

- Not to be used on the floor.
- Minimum of 8 foot ceiling to suspend.
- Minimum of 6 foot clear distance from walls and other obstacles.

### HEIGHT ADJUSTER OR SAFETY ROTATIONAL DEVICE NEEDED:

- A Southpaw Height Adjuster, item #5000 may be needed to adjust your equipment to the correct height for clients. Suspension from two points is recommended to allow for head room. Suspension point should be approximately 3' apart.
- If hanging from one suspension point a Southpaw Safety Rotational Device, item #2000 must be used to prevent damage to your equipment, except where otherwise specified.

**WORKING LOAD:** 200 LBS.

### TO USE:

Insert deflated ball into cover through Velcro closure on top of cover. Inflate ball slowly through opening and adjust cover as needed.

The Moon Ball Swing is to be used suspended. For gravitationally insecure clients you may adjust the Moon Ball Swing closer to the floor. Use caution not to snag the fabric.

### SUSPENSION PROCEDURES:

The Moon Ball Swing comes with VSDs attached to each side of the swing. To suspend your Moon Ball Swing, connect the Locking Safety Snap at the top of the VSD to your suspension point(s).

### **MAINTENANCE CHECKLIST:**

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the Locking Safety Snap for wear. If wear is more than 30% of the diameter of the snap, discontinue use. (See Safety Snap Instruction Sheet).
- Check the gate on the Locking Safety Snap to make sure that it is working properly. If not working properly, discontinue use.
- Check the Velcro to make sure that it will close completely. If Velcro becomes separated, discontinue use until repaired.
- Check all seams for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.
- Check the top of the swing where the Locking Safety Snap goes through for wear. If wear is observed, discontinue use until repaired.

### **STORAGE PROCEDURES:**

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

### **CLEANING INSTRUCTIONS:**

- Check for client allergies first.
- Cover is machine washable.
- Air dry or dry in dryer using *low* heat.
- Wipe ball with cloth and disinfectant.

## **CAUTION**

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Please visit us @ [www.southpaw.com](http://www.southpaw.com)  
© 2019 Southpaw Enterprises, Inc.