



# SOUTHPAW®

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

## Instruction Sheet Standard Bolster Upper Works 121000



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises

### MATERIALS LIST:

- 1 - Safety Rotational / Rope Assembly
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

**INSTALLATION: Note: Rope sizes have changed to accommodate new bar design (121000 only). The updated model will work with older bars. To use remove safety snap from center loop, and allow to hang loose.**

- Disconnect your swing from the suspension point.
- Remove the center and side ropes from the safety snaps.
- Connect new ropes to safety snaps and hang swing.

### MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed

- Check safety rotational device for wear on the safety snap and eyebolt. If found on eyebolt, discontinue use until repaired. If found on safety snap, discontinue use until replaced.
- If device does not spin smoothly or makes noise discontinue use.
- Check ropes for fraying (be sure to check end of rope under black boot).
- Check wire loops for broken or split wires discontinue use if found.
- Check all Safety Snaps for wear. Refer to the Safety Snap Instruction Sheet for detailed maintenance checklist.
- Check ropes for loose or missing metal rope clips. If found discontinue use.

### STORAGE PROCEDURES:

- Store out of Direct Sunlight
- Store where not accessible by clients without proper supervision

### CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Wash using a damp cloth with mild soap and water.
- Do not use abrasive cleaners.

## **CAUTION**

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Please visit us at [www.southpaw.com](http://www.southpaw.com)  
© 2019 Southpaw Enterprises, Inc.