



Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

## Instruction Sheet

### Advantage Line Infant Adaptation Kit 121 820



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment. If this Instruction Sheet is misplaced, please call or write for a replacement. (See address below).

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

**Warning:** Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

#### MATERIALS LIST:

- 1 - Advantage Line Infant Adaptation Kit
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

The Advantage Line Infant Adaptation Kit is compatible with Southpaw's Advantage Line Platform Swing 120040: Modified Platform Swing 1805: and Standard Platform Swing 1800.

**(Remember: Southpaw Safety Rotational Device must be used or damage to the equipment will result).**

**WARNING: Never leave an infant unattended in the swing. The sides of the Infant Adaptation Kit are meant only to keep an infant from rolling out of the swing when in use. It is possible that an infant could crawl or fall over the side without proper supervision. The sides are not designed to be used in a chair back fashion. After sides are in the closed position, test by pulling on them to put them under stress. If they open easily, reposition fasteners so that they hold properly. Always use appropriate matting under and around the swing.**

#### ASSEMBLY PROCEDURE:

- Place the Infant Adaptation Kit on the Platform Swing between the four ropes with the folded flaps and hook fasteners facing up. The ropes will fit into the notches in the corners of the Infant Adaptation Kit.
- Notice that two opposing sides will have folded extension flaps with 2" hook fasteners. The other two opposing sides will have 4" hook fasteners.
- Separate the folded extension flaps with the 2" hook from the material. Raise the side and wrap the extensions on each side around the platform swing rope. Press the hook fastener into the material to secure the extensions around the rope. Repeat this procedure on the opposing side. This will give you two sides of the kit.
- On the other two opposing sides, separate the 4" hook flaps from the material. Raise the side up and wrap the hook fastener around the corner formed by the rope and press the hook fastener into the material of the side attached to the rope. Repeat on all four corners.
- This completes the assembly.

#### MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.

**STORAGE PROCEDURES:**

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

**CLEANING INSTRUCTIONS:**

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Machine wash cold water and mild detergent. Air dry.

**CAUTION**

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401-1047  
Toll Free - (800) 228-1698  
Please visit us at [southpaw.com](http://southpaw.com)  
© 2020 Southpaw Enterprises, Inc.