



southpaw enterprises, inc.®

Instruction Sheet Linear Glider Tent Cover 121855

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

1 - Linear Glider Tent Cover 1 - Instruction Sheet 1 - Maintenance Checklist

Hanging the Linear Glider Tent Cover

- 1) There is one loop at each end that attaches to the safety snap holding the Linear Glider. Tent cover should be on the outside of the ropes supporting the Linear Glider.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the webbing for wear at the point the webbing rubs against the Safety Snap. If wear is found, discontinue use until repaired.
- Check the corners of the Linear Glider Tent. These areas are critical. If the stitching is coming loose, or the material is torn or pulling out of the corner, discontinue use until repaired.
- Check seams for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.

MATTING PRECAUTIONS:

- Always use appropriate matting.
- Crash mats and wall mats must be used with all hanging equipment.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Machine wash warm water, mild detergent. Tumble dry, low heat or air dry.

CAUTION

- ◆ Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.
- ◆ Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.®**

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the web at <http://southpawenterprises.com>

