



Instruction Sheet

2-in-1 Frog Swing 121920

Store these instructions in a safe place with the enclosed maintenance checklist. You may also access them on our website, southpaw.com.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

MATERIALS LIST

- 1 - 121920 Frog Swing (XB1927)
- 2 - Safety Snaps (0100)
- 1 - Instruction Sheet
- 1 - Safety Snap Instruction Sheet
- 1 - Maintenance Checklist

Warning: Failure to properly install a Southpaw forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the Southpaw forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Minimum height requirement is 8 feet.
- Caution must be used to keep a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

- Use of the Southpaw Height Adjuster (5000) is recommended for ease of adjusting your equipment.
- A Southpaw Safety Rotational Device (2000) must be used for maximum effectiveness and decreased stress on equipment.

WORKING LOAD: 120 pounds. (Up to 150 pounds with additional loops.)

HANGING PROCEDURES

IMPORTANT: KEEP ALL PARTS TOGETHER

Attach your Frog Swing to your ceiling support system using the Safety Snap(s) provided. The Frog Swing is now ready for use.

MAINTENANCE CHECKLIST

All of the following wear points should be check on a monthly basis and recorded on the checklist enclosed.

- Safety Snaps — see enclosed Safety Snap Instruction Sheet
- Stimulation Loops — Check for loose stitching around the "ball" near the top and the bottom, and holding the loops on the top and bottom. If found discontinue use until repaired or replaced.
- Ropes — Check for fraying rope. If found, discontinue use until repaired.
- Rope splice-clips — These are the small metal rings found near each splice of the white poly-rope. If any are found to be loose or missing, discontinue use.
- Swing Seat — Check for worn loops or tears around carpet. Caution: The carpet backing contains latex. If the carpet backing becomes exposed, discontinue use. Check clients for allergies.
- Check wood for cracks or splinters. If cracked, discontinue use immediately. For splinters, sand smooth and refinish with clear lacquer.

MATTING PRECAUTIONS

- Appropriate mats must be used at all times.

STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible without proper supervision.
- Keep away from all chemicals and household cleaners.

IMPORTANT: KEEP ALL PARTS TOGETHER.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Carpet may be cleaned with dry carpet shampoo.
- All other parts may be cleaned with damp cloth.
- DO NOT IMMERSE IN WATER

CAUTION

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us southpaw.com

© 2025 Southpaw Enterprises, Inc.