



southpaw enterprises, inc.®

Instruction Sheet

126016: Carpet Kit

Home Therapy System Platform Swing

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- 1 - Carpet for the Home Therapy System Platform Swing
- 4 - Adhesive loop squares
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

Directions

Your carpet comes with the four adhesive loop squares attached to the hook squares on the bottom of the carpet.

1. Place the platform swing on the floor. Place the carpet on top of the platform swing, centered between the ropes and square with the edges of the platform.
2. Once the carpet is in place and square, kneel on the carpet to hold it in place. Turn one corner up, revealing the hook and loop squares. Peel the backing off of the adhesive loop square, leaving it attached to the hook square. Carefully lay the corner of the carpet back down, and press firmly on top of the corner.
3. Repeat step 2 with the remaining corners. Your carpet is now ready to use.


NOTE: To make sure that the adhesive squares never release from the board, you can staple them in place. Be sure to use 3/8" staples, so that they do not go all the way through the wood. Staple them according to the pattern to the right.

STORAGE PROCEDURE

- Store where not accessible by clients.
- Store out of direct sunlight.

CLEANING INSTRUCTIONS

- Vacuum carpet.
- Carpet may be cleaned with foam carpet cleaner.
- Carpet is not machine washable.

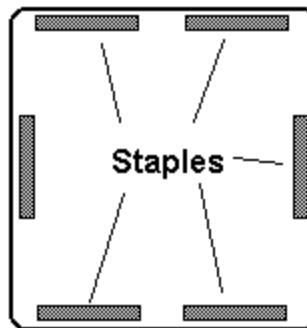

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.®**