



Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet

Sensory Snake 140010 Small



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

140010 Small

- 1 - Sensory Snake 2 lbs. weight
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

INSTRUCTIONS FOR USE:

Southpaw's Sensory Snake provides proprioceptive feedback on the neck and shoulders. Weight can be adjusted by shifting the weight of the beans inside the snake (heavier in front or back).

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.

STORAGE PROCEDURES:

- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Clean with a damp cloth. Air dry.
- **DO NOT MACHINE WASH.**
- **Note:** Washable covers for the Sensory Snakes are available, product numbers 140011 small and 140016 large.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047
Toll Free - (800) 228-1698
Please visit us at www.southpaw.com
© 2019 Southpaw Enterprises, Inc.