



southpaw enterprises, inc.®

Instruction Sheet: Balance Pad Large #150161

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please read this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST #150161

- 2 - Floor Half Roll
- 1 - Instruction Sheet
- 1 - Maintenance Checklist
- 1 - Cover for Balance Pad

SPECIAL NOTES:

- Use caution if using Floor Roll with more than one client at a time.
- Do not use with any ramp or platform.
- Although the roll is made of foam, use caution that fingers do not get pinched where the roll meets the floor.
- The foam used in the Floor Roll is constructed of very sturdy foam, however if it is flattened or deformed severely, it may not recover its original shape.

WORKING LOAD: 225 lbs.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the vinyl cover for loose stitches, rips, or tears. If loose stitches, rips, or tears are found, discontinue use until repaired. CAUTION: Do not allow clients to pick foam from inside and ingest it.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- It is best to store barrel in an upright position.
- Do not store near chemicals or household cleaners.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Wipe with a damp cloth using soap and water or disinfectant.

ASSEMBLING INSTRUCTIONS:

- Unzip the two ends of the vinyl cover, insert one of the half round foams into one side of the cover. Leave this side un-zipped for now as it will help air escape while putting the other foam in. (The flat part of the foam should be on the Orange side)
- Put the other half round foam into the other end of the cover. Zip both sides up now.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.®**

© 2019 Southpaw Enterprises, Inc.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047
Toll Free - (800) 228-1698
Please visit us at www.southpaw.com