



# Instruction Sheet

## Interaction Dual Swing 1630

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### MATERIALS LIST

- 1 - Completely assembled swing with Safety Snap
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

### MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

Minimum height required is 8 ft.

Caution must be used to keep a clear distance of ft. from walls and other obstacles.

### HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

Use of a Southpaw Height Adjuster (# 5000) is recommended for ease of adjusting your equipment.

A Southpaw Safety Rotational Device (# 2000) *must* be used to allow the Interaction Dual Swing to spin properly and prevent damage to your swing.

**WORKING LOAD:** 400 lbs.

**HANGING PROCEDURE:** Connect the safety snap attached to the top of the swing to your Southpaw Safety Rotational Device and then to your suspension system.

### MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check ropes for loose or missing metal rope clips. If loose or missing clips are found, discontinue use until repaired.
- Check wood bar for cracks or splinters. If cracked, discontinue use immediately. For splinters, sand smooth.
- Check ropes for wear especially at the top, bottom, and at the bar. If excessive wear is found, discontinue use until repaired.
- Check loops for wear. If excessive wear is found, discontinue use until repaired.

**MATTING PRECAUTIONS: Always use appropriate matting.**

### **CLEANING INSTRUCTIONS**

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Loops may be hand washed in warm water and mild soap or detergent. Drip dry.
- Do not clean ropes.

## **CAUTION**

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Please visit us @ [www.southpaw.com](http://www.southpaw.com)  
© 2019 Southpaw Enterprises, Inc.