



# SOUTHPAW®

Store these instructions in a safe place or with the enclosed maintenance checklist

## Instruction Sheet

### 165020 Tactile Bolster Swing Covers



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT:** Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

#### MATERIALS LIST

165020

- 1 - Tactile Fleece Bolster Swing Cover
- 1 - Tactile Velour Bolster Swing Cover
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

#### ASSEMBLY

- Feed rope from one side of the Bolster Swing through entire cover. Slide cover over body of swing.

#### MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.

#### STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

#### CLEANING INSTRUCTIONS

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Covers may be machine washed using cold water and mild detergent.
- Tumble dry, **low** heat.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at [southpaw.com](http://southpaw.com)

© 2021 Southpaw Enterprises, Inc.