



**southpaw enterprises, inc.®**

Store these instructions in a safe place or with the enclosed maintenance checklist

# Instruction Sheet

## Infant Adaption Kit 1820



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT:** Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

### MATERIALS LIST:

- 1 - Infant Adaption Kit
- 1 - Mattress
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

**Note:** Must be used with Southpaw Platform Swing (not included).





**(Remember: Southpaw Safety Rotational Device must be used or damage to the equipment will result).**

**WARNING:** Never leave an infant unattended in the swing. The sides of the Infant Adaptation Kit are meant only to keep an infant from rolling out of the swing when in use. It is possible that an infant could crawl or fall over the side without proper supervision. The sides are not designed to be used in a chair back fashion. After sides are in the closed position, test by pulling on them to put them under stress. If they open easily, reposition fasteners so that they hold properly. Always use appropriate matting under and around the swing.

### ASSEMBLY PROCEDURES:

1. Place the Infant Adaption Kit on the Platform Swing between the four ropes. The ropes will fit into the notches in the corners of the Infant Adaption Kit. (See figure 1.). Sides "B" and "D" have Southpaw labels. (These sides are to be used as "doors").

**Warning:** Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

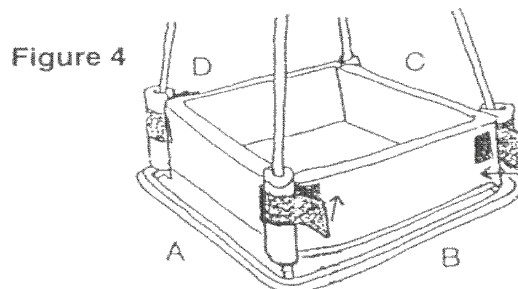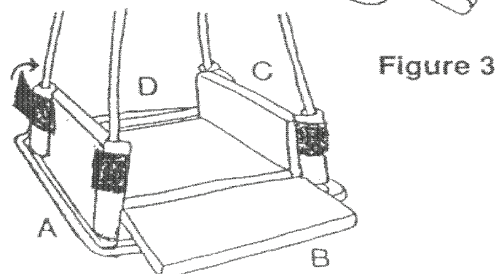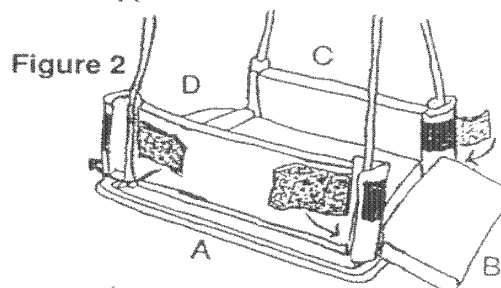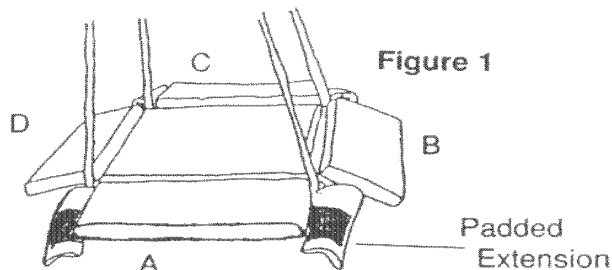

2. Wrap padded extension around white Platform Swing rope, tuck tightly, then wrap the velcro flap around the padded extension to hold it in place. The velcro flap should wrap completely around the padded extension so that the "prickly" side is completely attached. **Note:** Sides "A" and "C" are secure in place. Ends "B" and "D" are still lying flat.
3. To secure sides "B" and "D", pull the velcro flap back about half way and flip up the sides (like the tailgate on a truck).
4. Now attach the flaps to the outside of sides "B" and "D". Velcro must be attached securely!
5. Place mattress inside.

The Southpaw Infant Adaption Kit is now ready to use.

#### **MAINTENANCE CHECKLIST:**

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- All seams and joints should be inspected regularly for postears. The batting is both polyethylene and urethane foam. If the foam becomes exposed, discontinue use until repaired.
- Hook and loop fasteners should be clean at all times to insure maximum closure strength. A stiff brush will help in cleaning.
- If hook and loop fasteners become loose from the Infant Adaption Kit, discontinue use immediately until repaired.

#### **STORAGE PROCEDURES:**

- Store where not accessible by clients without proper supervision..
- Store out of direct sunlight.

#### **CLEANING INSTRUCTIONS:**

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Wipe clean with a damp cloth using soap and water or disinfectant.
- Dry before using.

#### **SUGGESTED USES:**

May be used as a crawling device when placed on floor (never use for crawling when suspended).

Crawling clients will want to gravitate towards the exposed ropes and pull on them. Use extreme care if using for this activity.

### **CAUTION**

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698