



SOUTHPAW®

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet First Cut™ Adapted Scissors 190040



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- 1 - Pair of Adapted Scissors
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

CAUTION: This product contains small parts that, if removed, could be a choking hazard for small children. This product should only be used with adult supervision.

INSTRUCTIONS FOR USE:

- Check the fit of the thumb cap with the thumb of the client or child. The scissors come assembled for use. If thumb cap is too tight on the child's hand, the cap can be removed completely for larger hands.
- The thumb cap may be inserted from either side making the scissors right or left handed.
- Loosen the finger cord and have the user grasp the scissors with the thumb in the thumb cap and fingers in the finger opening with the index finger in the loop of cord on the outside of the handle.
- Gently pull the end of the elastic cord until it is snug on the users finger. This will allow the user to maintain a good grip and control of the scissors.
- The two halves of the scissors are held together by a screw. If you want to increase the effort required to operate the scissors (for children with poor sensory modulation, or to work on strength), you can tighten this screw. Likewise, you can loosen the screw to make scissor operation easier. Note that loosening the screw too much will prevent the scissors from working correctly.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check plastic handles for cracks. If cracks are found, discontinue use immediately.
- Check that the elastic cord is firmly attached to the handle and the vinyl cap to the end of the cord. If either one becomes detached, they can be reattached with super glue (cyanoacrylate).

STORAGE INSTRUCTIONS:

- Store where not accessible without adult supervision.

CLEANING PROCEDURES:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Wipe with a damp cloth using mild soap or disinfectant.
- Dry before using.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us at southpaw.com

© 2024 Southpaw Enterprises, Inc.

