

Instruction Sheet Frog Swing 1925

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

MATERIALS LIST

- 1 1925 Frog Swing
- 3 Stimulation Loops
- 2 Locking Safety Snaps
- 1 Instruction Sheet
- 1 Safety Snap Instruction Sheet
- 1 Maintenance Checklist

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

- Minimum height requirement is 8 feet.
- Caution must be used to keep a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

- Use of the Southpaw Height Adjuster (5000) is recommended for ease of adjusting your equipment.
- A Southpaw Safety Rotational Device (2000) must be used for maximum effectiveness and decreased stress on equipment.

WORKING LOAD: 35 - 100 pounds. (Maximum 160 pounds with additional loops.)

HANGING PROCEDURES

IMPORTANT: KEEP ALL PARTS TOGETHER

Attach your Frog Swing to your ceiling support system using the Locking Safety Snap provided. The Frog Swing is now ready for use.

MAINTENANCE CHECKLIST

All of the following wear points should be check on a monthly basis and recorded on the checklist enclosed.

- Locking Safety Snaps see enclosed Safety Snap Instruction Sheet
- Stimulation Loops Check for loose stitching around the "ball" near the top and the bottom, and holding the loops on the top and bottom. If found discontinue use until repaired or replaced.
- Ropes Check for fraying rope. If found, discontinue use until repaired.
- Rope splice-clips These are the small metal rings found near each splice of the white poly-rope. If any are found to be loose or missing, discontinue use.
- Swing Seat Check for worn loops or tears around carpet. Caution: The carpet backing contains latex. If the carpet backing becomes exposed, discontinue use. Check clients for allergies.
- Check wood for cracks or splinters. If cracked, discontinue use immediately. For splinters, sand smooth and refinish with clear lacquer.

MATTING PRECAUTIONS

• Crash mats and side mats must be used at all times.

STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible without proper supervision.
- Keep away from all chemicals and household cleaners.

IMPORTANT: KEEP ALL PARTS TOGETHER.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- All parts may be cleaned with damp cloth.
- DO NOT IMMERSE IN WATER

CAUTION

- Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.
- ♦ Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047
Toll Free - (800) 228-1698
Please visit us at southpaw.com
© 2021 Southpaw Enterprises, Inc.