



Instruction Sheet

Adjustable Scooter Board 1950

Store these instructions in a safe place along with the enclosed maintenance checklist. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

MATERIALS LIST (If any of these materials are missing, please call or write Southpaw immediately.)

1 - SCOOTER BOARD
1 - PULL STRAP

1 - INSTRUCTION SHEET
1 - MAINTENANCE CHECKLIST

WORKING LOAD: 150 POUNDS

IMPORTANT: KEEP ALL PARTS TOGETHER

ADJUSTMENT PROCEDURES

To adjust the length of your Scooter Board lay it on its top. Lift the latch key to a right angle, (See fig. 1.) Turn this latch key counter clockwise to loosen the latch enough to be lifted. (See fig. 2.) This will allow you to pull the sections far enough apart to remove or replace the center section. Simply push the sections completely together and place the latch back down in the hook, turn the latch key clockwise to tighten.

NOTE: A pull loop is attached to the front underside of the Scooter Board so that the client may be pulled along by the pull strap provided. (See fig. 3).

MAINTENANCE CHECKLIST

✓ All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check tow strap for loose or missing stitches. If loose or missing stitches are found discontinue use.
- Check tow strap swivel hook to make sure it is functioning properly, if it is not discontinue use.
- Check latches to make sure they are securely attached and working properly. If not discontinue use and contact Southpaw.
- Check cover for rips and exposed foam. If foam is showing, repair or discontinue use to prevent clients from ingesting foam.
- Check casters to make sure all are rolling properly. If casters are not rolling properly contact Southpaw.

MATTING PRECAUTIONS

- Always use appropriate matting.

STORAGE PROCEDURES

- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean with a damp cloth using soap and water or disinfectant.
- Dry before using.
- Do not immerse in water

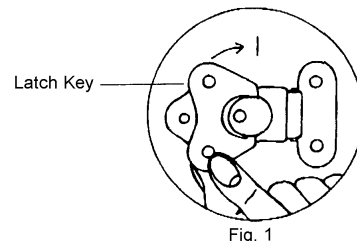


Fig. 1

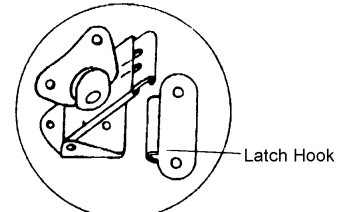


Fig. 2

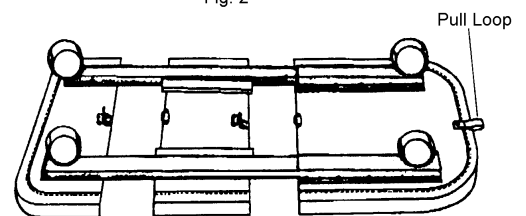


Fig. 3

CAUTION

- Eliminate obstructions that may be on the floor
- No standing on the Scooter Board
- Only one client at a time
- Avoid riding stairs
- Avoid riding backwards

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision. **This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047 • Toll Free - (800) 228-1698
Please visit us at southpaw.com
© 2024 Southpaw Enterprises, Inc.



EQUIPMENT MAINTENANCE CHECKLIST

Inspection Action: (I) Inspect (R) Replace

[illegible]

* This sheet is for example only. It is not meant to, in any way, replace county, state, or federal standards.