

Information for
Item # 220020
Awesome Hammock

Awesome Polyester Suspension Therapy Swing

Cleaning Instructions

Unscrew the metal Quick Link just enough twists to slide the white neo-cord bridal line from the Quick Link. The net is now ready to wash.

The metal Quick Link and the polyester Suspension Therapy Swing remain TOGETHER for easy detachment and re-attachment.

Please do NOT remove the Quick Link from the Eye Loop.

WASHING OPTIONS

You may now dip the polyester net in your disinfectant solution.

Please do NOT put the loops and the metal Quick Links in the disinfectant solution.

Keep them OUT of the solution.

Rinse the polyester net thoroughly in hot water.

NOT USING A DISINFECTANT SOLUTION?

If you are NOT using a disinfectant solution, wash the polyester swing in hot water with detergent. Never use antibacterial detergent. The agents in this product are harmful to the human body.

You may use a mild bleach making sure that the polyester net is rinsed thoroughly in hot water to avoid the "bleach" odor.

Scrub soiled areas as necessary.

Rinse in hot water.

DRYING OPTIONS: Drying Time Ranges from 20 to 25 minutes

Swing dry and/or pat dry with a towel.

You may also re-attach the metal Quick Link to the white neo-cord and place a long stick (clean broom for example) inside the net allowing more surface area exposed to the air.

Any Questions? Please call us at (208) 383-9636

Email: awesomehammocks@yahoo.com www.alphawaveengineers.com


AWESOME SUSPENSION SWING THERAPY

SUSPENSION SWING THERAPY NATURALLY ACTIVATES ALPHA WAVE BRAIN STATE WHICH IN-TURN INCREASES ENDORPHIN SECRETION PROMOTING RELAXATION AND CALM THOUGHTS

BOOSTING THE IMMUNE SYSTEM. Call: (208) 383-9636

Email: awesomehammocks.com

www.alphawaveengineers.com

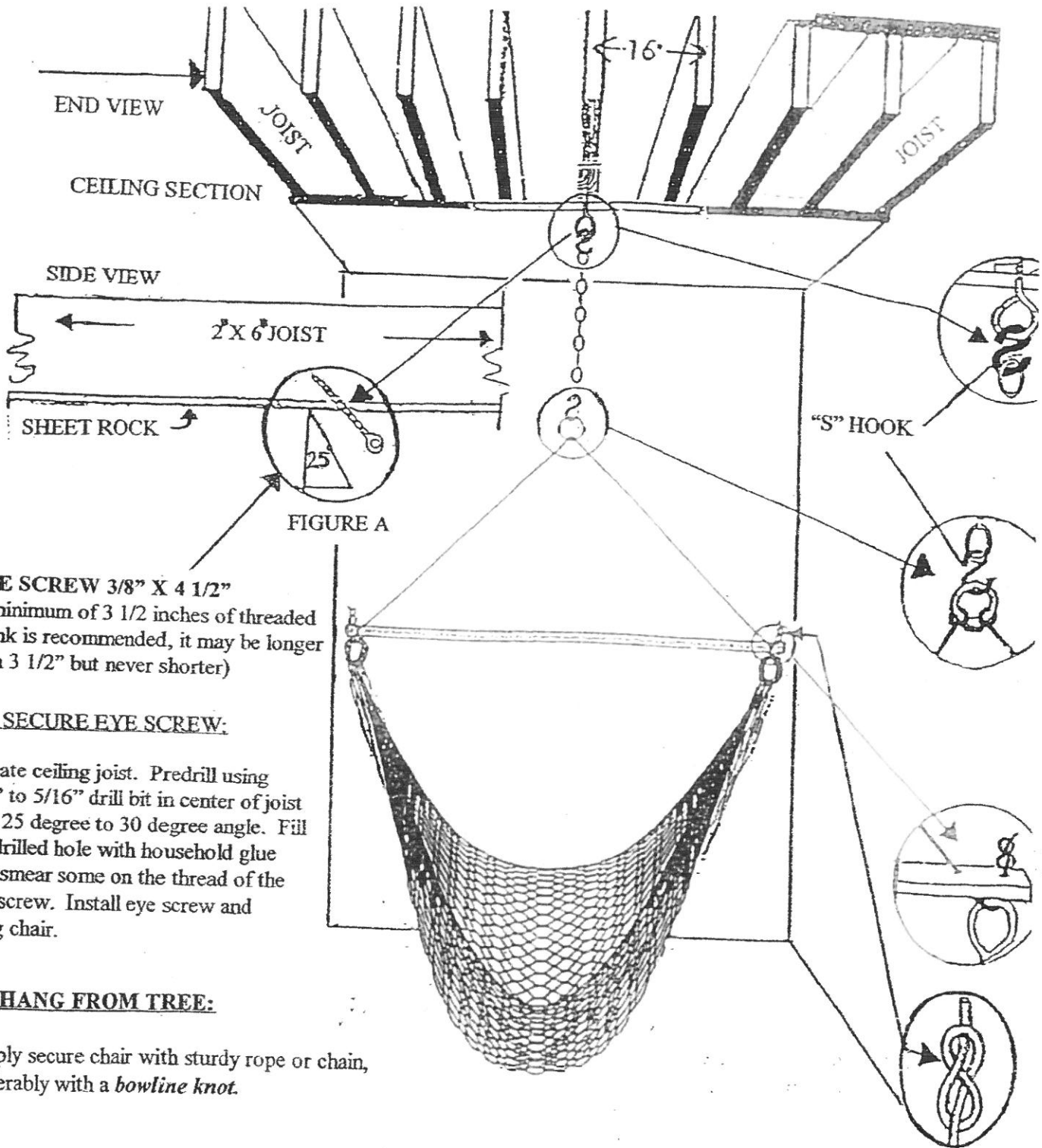

EYE SCREW 3/8" X 4 1/2"

(a minimum of 3 1/2 inches of threaded shank is recommended, it may be longer than 3 1/2" but never shorter)

TO SECURE EYE SCREW:

Locate ceiling joist. Pre-drill using 1/4" to 5/16" drill bit in center of joist at a 25 degree to 30 degree angle. Fill predrilled hole with household glue and smear some on the thread of the eye screw. Install eye screw and hang chair.

TO HANG FROM TREE:

Simply secure chair with sturdy rope or chain, preferably with a *bowline knot*.

"FIGURE 8 KNOT"