



# SOUTHPAW®

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

## Instruction Sheet

## C-Stand Replacement Height Adjusting Strap 227100



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT:** Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

**Warning:** Failure to properly install a Southpaw forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the Southpaw forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### MATERIALS LIST:

- 1 - Loop Strap
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

### TO INSTALL THE STRAP:

- Remove the old loop strap from the eye bolt.
- Hold the loop strap by the end with the top loop (all the other loops hanging down).
- Put the top loop of the strap through the eye bolt (fig. 1)
- The top loop is made with a half twist. One side of the loop will be larger than the other. Turn the loop so that the largest side of the loop is toward the back (flat side) of the strap. It will look like a funnel (fig. 2).
- Put the bottom end of the strap through the top loop from the back side (large opening of the funnel) and pull the strap tight around the eye bolt (fig. 3).
- As the strap tightens around the eye bolt, adjust the top loop so that it looks like a necktie knot (fig. 4).

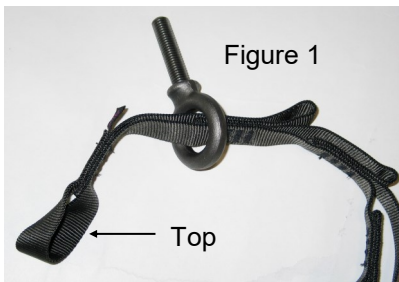


Figure 1



Figure 2

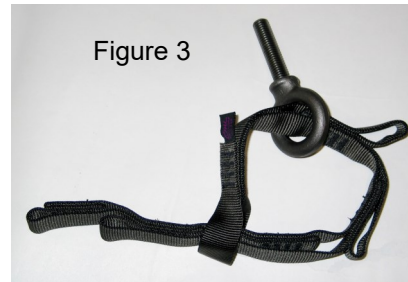


Figure 3

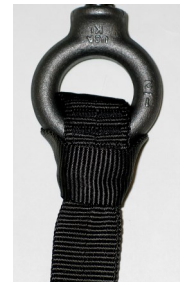


Figure 4

### MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the strap for fraying at the eye bolt contact point. If fraying is found, discontinue use and replace.
- Check attachment loops on strap for fraying. If fraying is found, discontinue use and replace.
- Check for loose or missing bar tack stitches on each loop. Discontinue use if loose or missing stitches are found.

**CAUTION:** Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear. Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision. **This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. **Be aware** - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401-1047 • Toll Free - (800) 228-1698 • Please visit us at [southpaw.com](http://southpaw.com)  
© 2024 Southpaw Enterprises, Inc.



# EQUIPMENT MAINTENANCE CHECKLIST

Inspection Action: ( I ) Inspect ( R ) Replace

[illegible]

\* This sheet is for example only. It is not meant to, in any way, replace county, state, or federal standards.