



Instruction Sheet

Portable Bubble Tube 303126



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

-Wooden Base Platform	-LED Remote Control	-Acrylic Tube
-Algae Preventer Solution	-4 Screws	-Allen Wrench

A. INSTRUCTIONS FOR ASSEMBLY

Remove small lid on top of the Platform. Place Bubble Column on top of the wooden slats positioning the holes over the pre-drilled holes on the slats so that the brass fitting is positioned on the same side as the white L-Tech box. Insert the 4 screws into holes in bubble column base and hand tighten screws with Allen wrench to secure to wooden platform support slats. Do not over tighten the screws. Connect hose from the base of the Bubble Column to the white plastic 1/4" stem attachment on the air pump.

USE DISTILLED WATER ONLY TO FILL BUBBLE COLUMN

Column takes just under 2 gallons of distilled water. Pour entire contents of Algae Preventer Solution when you fill the column. **DO NOT OVERFILL.** When pouring water into the Column plug the unit in so the air pump is running. This will let you know the maximum water level when the unit is running. Replace lid.

B. LED SETTINGS

The Unit comes pre-set from the factory with the M3 setting. A Remote Control is provided if you want to change the setting. Changes should be made by the Therapist ONLY.

-REMOTE OPERATION-Explanation of Controls

MODE-Press to change to next Mode

PAUSE – Press to make LED Color stay at present; press again to move on.

SPEED UP- Press to change speed of color change faster.

SPEED DOWN – Press to change speed of color change slower.

MODES

M1-Seven Color Flash

M2-Leaning Color Fade

M3- Full Color Fade

M4-Loop

M5-11 – Seven Color Fixation

An instruction from the LED manufacturer is also included with the Remote.

C. CLEANING / INSPECTION / MAINTENANCE

Recommended cleaning solution is a 50/50 mixture of Hydrogen Peroxide and water and/or a small amount of mild detergent. Use a soft cotton cloth in conjunction with any cleaner.

DO NOT USE ANY CLEANING PRODUCTS CONTAINING ISOPROPYL OR ETHYL ALCOHOL. DO NOT USE ANY AEROSOL SPRAYS AS THEY ALL CONTAIN ALCOHOL. DO NOT USE ABRASIVE CLEANERS, AS THEY WILL SCRATCH. DO NOT USE PAPER TOWELS AS MAY SCRATCH THE BUBBLE TUBE.

D. MAINTENANCE

BUBBLE TUBE – Only Distilled Water should be used in the tube for operation. The water should be replaced every three (3) months with fresh Distilled Water. To drain disconnect air line from the base of the tube, remove the four mounting screws and lift out the tube to drain water. Return tube to base, secure mounting screws and reattach air line to base of tube and fill with fresh water. The Bubble Tube holds slightly less than three (3) gallons of water. Fill the Bubble Tube with the pump running to the label near the top of the tube marked “Do Not Use Alcohol to Clean” to be sure not to overfill. When adding new water 2 ounces of diluted, or a few drops of concentrated anti-algae preventer should be added to the water.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



FORM CODE: XI- 303126 © 2019 Southpaw Enterprises, Inc.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at
<http://southpawenterprises.com>