



Instruction Sheet

Vibro-Acoustic Pillow - 303206

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

Vibro-Acoustic Pillow

INSTRUCTIONS FOR ASSEMBLY

The Vibro-Acoustic Pillow comes out of the box completely assembled. In order to make the Pillow vibrate the product must be connected to the Southpaw Radio Rover (303100), Mini Rover (303600) or Wall Stereo (303900). Use with any other stereo equipment will void the warranty.

CLEANING / INSPECTION / MAINTENANCE

Recommended cleaning solution is a 50/50 mixture of Hydrogen Peroxide and water, an all purpose cleaner without bleach and/or a small amount of mild detergent. Use a soft cotton cloth in conjunction with any cleaner.

MAINTENANCE

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for torn vinyl and loose or ripped stitches. If tears or loose stitches are found, discontinue use until repaired.
- Check for loose or frayed speaker wires. If frayed wires are found, discontinue use until repaired.

STORAGE PROCEDURES

- Store where not accessible by clients without proper supervision.

CARE SHOULD BE TAKEN TO REMOVE ALL SHARP OBJECTS FROM POCKETS, AS WELL AS ANY METAL BRACES, PRIOR TO USING THE VIBRO-ACOUSTIC PILLOW.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Please visit us @ www.southpaw.com
© 2019 Southpaw Enterprises, Inc.