



# Instruction Sheet Vibro Adaptor for Wheelchairs

**303250 - Single**  
**303251 - Double**

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

## MATERIALS LIST:

- 1 - Vibro Adaptor assembly for Wheelchair —comes pre- assembled and includes wheelchair wrap, pouch, and vibro unit
- 1—18 ft RCA extension cable
- 1 - Instruction Sheet      1 - Maintenance Checklist

In order to make the Vibro Adaptor vibrate, the product must be connected to a Southpaw Radio Rover (303100,), Mini Rover (303199), Wall Radio (303900) or Corner Radio (303910). Use with any other stereo equipment will void the warranty.

## INSTRUCTIONS FOR USE:

**Single Vibro Adaptor** - Place the wheelchair wrap snugly around the back support for the wheelchair, with the speaker pocket behind the chair, and fasten with the hook and loop fastener. Connect the wired end connector to one end of the 18 ft RCA cable provided. Connect the opposite end of the 18 ft RCA cable to one of the Vibro ports on your Southpaw Stereo radio product. Power on Stereo unit and follow the "Vibro-Acoustic use" section in the connected Southpaw Stereo product's instructions.

**Double Vibro Adaptor** - This consists of three pieces, the wheelchair wrap and two support straps. Each support strap has a hook fastener tab at the end that will attach to the wheelchair wrap. Place the wheelchair wrap snugly around the back support for the wheelchair, with the speaker pockets behind the chair and fasten with the hook and loop fastener. Attach the support straps to the front and back of the wheelchair wrap to help support the weight of the speakers. Connect the wired end connector to one end of the 18 ft RCA cable provided. Connect the opposite end of the 18 ft RCA cable to the Vibro port on your Southpaw Stereo radio product. Power on Stereo unit and follow the "Vibro-Acoustic use" section in the connected Southpaw Stereo product's instructions.

## MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.
- Check for loose or frayed speaker wires. If frayed wires are found, discontinue use until repaired.

## STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

## CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Remove speakers and speaker wires, then machine wash cold water and mild detergent. Air dry.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write

Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Visit us at [southpaw.com](http://southpaw.com)  
Toll Free (800) 228-1698  
© 2024 Southpaw Enterprises, Inc.