



SOUTHPAW®

Instruction Sheet

Wireless Vibro Bean Bag Chair 303255



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

Southpaw Vinyl Outer Cover (light blue)	Wireless Bluetooth Amplifier and Power Supply Cord
Fabric Inner Liner (tan color) and Foam	Vibro-Acoustic Foam Cube Assembly with cord & vibro end connectors
Instruction Sheet	Maintenance and Inspection Checklist

INSTRUCTIONS FOR ASSEMBLY

For assembly instructions, refer to Step-by-Step guide (with images) which is located **on the back of this instruction sheet**.

OPERATION:

In order to make the Bean Bag vibrate, it's amplifier unit must be paired with a Bluetooth-capable connection device—ie: cell phone, tablet, computer, etc (not included). The Bluetooth-capable device that will be used needs to have audio file playing capabilities. Audio/music files which include lower, "bass tone" sounds must be played in order to activate the vibration effects of the wireless bean bag chair —ie: sounds made by drums, a bass, a tuba, etc. The Bluetooth Amplifier's power supply must be plugged into a power outlet during operation.

PAIR / SYNC DEVICE to the WIRELESS BLUETOOTH AMPLIFIER

Pair/Sync your Bluetooth capable device (ie: phone or tablet) with the Wireless Bluetooth amplifier that came with the bean bag chair:

- Power on the Wireless amplifier device by single long press of On/Off button.
- When unit powers on there will be both a red and blue indicator light which will both be lit.
- Using your Bluetooth capable device, select the " DTA-21.BT2 " amplifier from the list on your device's wireless pairing menu options.
- When the Wireless Vibro Bean Bag's Bluetooth amplifier has been successfully paired to your device, a solid blue indicator light will appear to the right of the On/Off button.

Refer to the wireless Bluetooth Amplifier instructions manufacturer's manual (see pages 5 and 6) for full information regarding the details and all operations of the Bluetooth amplifier unit.

TURNING ON THE VIBRO-ACOUSTIC EFFECTS

Audio/music files which include lower, "bass tone" sounds **MUST** be played in order to activate the vibration effects in the wireless bean bag chair. (ie: sounds made by drums, a bass, a tuba, etc). After successfully pairing the Bluetooth wireless amplifier to your device, open the desired app on your device which will be used for playing the music, audio files, etc. Select the desired music or sound files and begin playing the files. Adjust the vibration effect's strength level as desired for end user. Please note that using music or sound file selections with lower toned "bass" or "drum-beat" type sounds within the music will create a stronger vibration effect.

ADJUSTING THE VIBRATION EFFECT STRENGTH LEVEL

The volume dial, sub woofer dial, and cross fade dials on the front of the wireless Bluetooth amplifier may all be adjusted in order to increase or decrease the "strength" of the vibration effects. Sound levels can also be adjusted by using your audio playing device's equalizer app or sound settings menu.

CLEANING / INSPECTION / MAINTENANCE / STORAGE

Cleaning Instructions: Check for client allergies first. The outer vinyl cover can be cleaned with any prescribed disinfectant. Dry before using. Care should be taken to remove all sharp objects from pockets, as well as any metal braces, prior to using the vibro-acoustic bean bag chair. Inspect covers and cords prior to use for any noticeable wear and tear such as tears, cuts, or loose connections. If any visible damages are observed, discontinue use until the damaged item (s) are repaired. Record all



SOUTHPAW® Assembly Instructions For Wireless Vibro Bean Bag Chair

Place the Southpaw outer vinyl liner over the inner fabric liner with shredded foam in it. Line up the zippers as shown in Image 1 at right.

1



Carefully open the inner liner zipper to expose the shredded foam inside and place the gray colored Foam Vibro Assembly into the shredded foam, see Image 2.

2



Slowly and gently bury the Foam Vibro assembly by nestling it into the shredded foam, see image 3. Carefully scoop a small amount of the shredded foam up and over top of the Foam Vibro Assembly while still keeping the shredded foam pieces inside the tan colored fabric inner liner.

Close the zipper on the fabric inner liner **leaving the wire with red and black plug ends on the outside, coming out of the end where zipper closes.**


3



Feed the red and black plug ends of the wire into the small opening at the bottom of the of the inside of the pocket area with Southpaw logo and then **gently** pull the wires out through the top opening of the pocket area, see image 4 at right.


Close the outer vinyl liner cover's zipper.

4

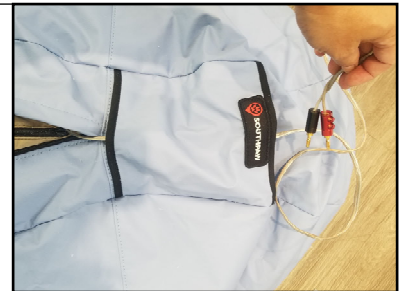

Attach the red and black plug ends (red to red / black to black) to the rear of the Wireless Amplifier unit. Insert the red and black plug ends into the pair of ports which are labeled as the "S" section connections—as shown in images 5 and 6 at right.

5

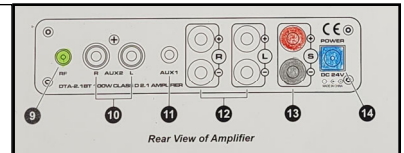


Insert the Amplifiers power supply, smaller adapter end into the DC power port on right rear of unit—as shown in images 5 and 6 at right. Place the wireless Bluetooth Amplifier inside the pocket the blue vinyl outer cover, orienting it so the on/off switch and the various adjustment dials on the unit can be easily accessed from the top opening of pocket.

6



The power cord may now be plugged into the nearest available electrical power outlet to provide power to the wireless amplifier unit. Bluetooth Pairing and operation instructions are listed on the front side of this instruction sheet.

7

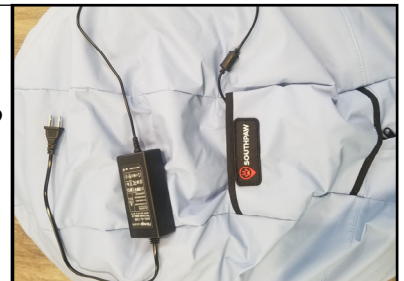

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



SOUTHPAW®

© 2024 Southpaw Enterprises, Inc.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at
<http://southpawenterprises.com>