



SOUTHPAW®

Instruction Sheet Wireless Vibro-Acoustic Rocker #303257



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.


MATERIALS LIST

Southpaw Vibro Rocker with Cover
Wireless Bluetooth Amplifier

Maintenance and Inspection Checklist

INSTRUCTIONS FOR ASSEMBLY

Assemble the Bluetooth wireless amplifier to the wire end connections as shown in image on right. Attach power adapter cord to the Bluetooth amplifier unit as shown in image at right. Power supply cord will need to be plugged into a power outlet for unit to operate.

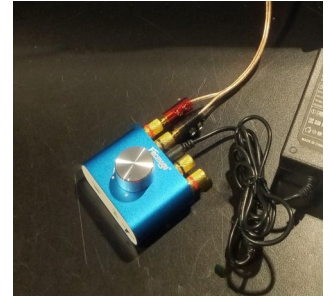

OPERATION:

In order to make the Rocker vibrate, it must be paired with a Bluetooth connection device (not included) which has audio file playing capabilities. To pair, on the audio playing device's pairing settings, search for the Bluetooth device called "F9000S" and pair/sync the audio playing device with the "F9000S" according to the directions of the Bluetooth-enabled audio player device.

After successfully pairing, select the desired music or sound files to play from the audio playing device. Note: Sound/Music selections with lower toned "bass" sounds will create more vibration effects. The sound volume may be adjusted using the audio playing device settings or using the adjustment dial on the wireless amp.

CLEANING / INSPECTION / MAINTENANCE

Cleaning Instructions: Check for client allergies first. The outer vinyl cover can be cleaned with any prescribed disinfectant. Dry before using.

Inspect covers and cords prior to use for any noticeable wear and tear such as tears, cuts, or loose connections. If any visible damages are observed, discontinue use until the damaged item (s) are repaired. Record all maintenance and inspection checks on the included checklist.

STORAGE PROCEDURES

Store where not accessible by clients without proper supervision.

CARE SHOULD BE TAKEN TO REMOVE ALL SHARP OBJECTS FROM POCKETS, AS WELL AS ANY METAL BRACES, PRIOR TO USING THE VIBRO-ACOUSTIC ROCKER.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



SOUTHPAW®

FORM CODE: XI- 303257 © 2020 Southpaw Enterprises, Inc.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at
<http://southpawenterprises.com>