



Instruction Sheet: Vibro-Acoustic Recliner #303260

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- 1 - Vibro-Acoustic Recliner
- 1 - 20' Speaker Wire

INSTALLATION INSTRUCTIONS :

The Recliner comes delivered completely assembled. In order to make the Recliner vibrate you must connect it to a Southpaw Radio Rover, Mini Rover, Wall or Corner Radio. Use with any other stereo equipment will void the warranty.

The speaker wire comes with RCA jacks at each end. Locate the speaker connection on the side of the chair and plug one end of the wire into the RCA socket. Connect the other end of the wire to the RCA socket on a Radio Rover.

MAINTENANCE:

Spot cleaning 1. Blot any excess liquid; wipe away excess mess with a dry, clean towel. 2. Mix together a simple soap and water solution. We recommend mixing 1/4 teaspoon enzyme laundry or dish detergent, like Tide®, Cheer® or Dawn®, with 1 cup warm water. 3. Apply the solution and agitate with a soft bristle brush. Make sure to brush lightly and work from the outside of the stain inward so as not to spread the stain. Rinse your sponge or brush frequently. 4. Blot with a clean towel and rinse. Repeat if necessary. 5. And remember, soap attracts dirt so make sure to rinse thoroughly. 6. Allow fabric to air dry.

Care should be taken to remove all sharp objects from pockets, as well as all metal braces or braces containing metal snaps or connectors, prior to sitting on the Recliner.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Visit us at southpaw.com