



southpaw enterprises, inc.®

Instruction Sheet

303491 Fiber Optic Tunnel with Vibro Floor Mat

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

1 - Fiber Optic Tunnel 1 - Vibro-Acoustic Floor Mat 2 - Adhesive Backed Velcro strips
1 - Instruction Sheet 1 - Maintenance Checklist

SPECIAL NOTES:

- Tunnel Arch Must be used with Fiber Optic Light Source (sold separately) - connect harness to light source.
- Vibro Floor Mat must be used with a Southpaw Stereo item (sold separately) to make the floor mat vibrate.
- Use caution if using Tunnel with more than one client at a time.
- Do not allow clients to stand on barrel. Do not use with any ramp or platform.
- Although the Tunnel is made of foam, use caution that fingers do not get pinched where the barrel meets the floor.
- The foam used in the Tunnel is constructed of very sturdy foam, however if it is flattened or deformed severely, it may not recover its original shape.

INSTRUCTIONS FOR ASSEMBLY:

- Turn the Fiber Optic Tunnel Arch so that it is oriented on its side so you can see the bottom of the arch.
- Attach (1) Velcro strip (do not remove adhesive backing) to one side bottom of Fiber Optic Tunnel Arch, matching it with the Velcro strip which has been sewn on to the Tunnel Arch during manufacturing. Repeat with the remaining Velcro strip on the second side of bottom of Tunnel Arch.
- Determine where the Tunnel Arch will be positioned on the Vibro-Acoustic Floor Mat. Remove adhesive backing from the Velcro strips attached to the bottom of the Arch.
- Holding Arch upright keep the Velcro adhesive from touching anything and then place Tunnel Arch onto the Vibro Mat in the location desired. Press down gently but firmly to allow the adhesive to adhere to the Vibro Mat

INSTRUCTIONS FOR USE:

- In order to make the Floor Mat vibrate the product must be connected to and used with a Southpaw Stereo item. Insert the RCA cord on the Vibro floor mat into the connection output jack on the Southpaw Stereo item. Attach an extension RCA cord and the cord connection adapter in order to lengthen the cord distance if your Southpaw Stereo is not near the Vibro floor mat. Follow directions included with the Southpaw Stereo item for Vibro-Acoustic use. Use with any other stereo equipment will void the warranty.
- Attach Tunnel arch's fiber optic harness connection to the Fiber Optic Light source and follow the instructions for the light source being used.

WORKING LOAD: 225 lbs.

CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Wipe with a damp, soft cloth using soap and water or disinfectant.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the vinyl cover for loose stitches, rips, or tears. If loose stitches, rips, or tears are found, discontinue use until repaired. CAUTION: Do not allow clients to pick foam from inside and ingest it.
- If vinyl should tear or rip, contact Southpaw to replace cover.
- There should be no need to get inside the Floor Mat to access the speakers.
- Check RCA cord and cord connection for any damage, loose, or bends.


CARE SHOULD BE TAKEN TO REMOVE ALL SHARP OBJECTS FROM POCKETS, AS WELL AS ANY METAL BRACES, PRIOR TO GETTING ON THE VIBRO-ACOUSTIC FLOOR MAT OR AROUND THE FIBER OPTIC TUNNEL.

STORAGE PROCEDURES:

- Store out of direct sunlight. Do not store near chemicals or household cleaners.
- It is best to store Fiber Optic Tunnel in an upright position.
- Store where not accessible by clients without proper supervision.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

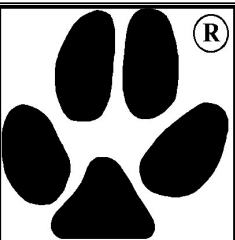

Southpaw Enterprises Inc.®

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698

Come see us on the World Wide Web at
<http://southpawenterprises.com>