



SOUTHPAW®

Instruction Sheet 33BPW

Battery Powered Bubble Cart



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- Wooden Cart
- Clear Acrylic Tube Column
- Remote Control for LED lights
- 4 Screws
- Allen Wrench
- Algae Preventer Solution

A. INSTRUCTIONS FOR ASSEMBLY

Wooden Cart will come assembled. Remove the two screws that secure the small lid on the platform and remove lid. Place Clear Acrylic Bubble Column on top of the wooden slats positioning the holes over the pre-drilled holes on the slats. Tighten screws with Allen wrench. Connect hose from the base of the Bubble Column to the air pump.

USE DISTILLED WATER ONLY TO FILL BUBBLE COLUMN

Column takes just under 2 gallons of distilled water. Pour half the contents of Algae Preventer Solution when you fill the column. **DO NOT OVERFILL.** When pouring water into the Column turn the unit on with the toggle switch on the side so the air pump is running. This will let you know the maximum water level when the unit is running. Insert fiber optic tails into the hole on the side of the box, attach harness to light source and tighten with set screw. Replace wooden cart lid. The cart's re-charge-able battery will need to be charged fully prior to the initial use of the cart. Allow the cart's battery to charge for 6-8 hours prior to use.

Important!

Do not use the unit while the battery is being charged as this can cause damage to the battery. Do not allow the unit's re-chargeable battery to become completely drained of it's power charge.

B. TO OPERATE THE CART:

Detach charging cord. Turn the black on/off power switch to the "On" position. LED lights, Fiber optic lights, and bubbles should start. Use unit as desired for up to 8-10 hours and then return to charging station to re-charge the battery. Do not use the unit while charging cord is plugged into the cart.

The battery for this unit will need to be re-charged every 8-10 hours of use or after any time the cart is not in use for more than two weeks. If the unit's lights begin to dim or the bubble flow rate slows, this is an indicator that the battery is beginning to lose its power charge and needs to be re-charged. If lights dim or bubbles slow, Turn the unit's power switch to the "Off" position and discontinue use of cart and re-charge by attaching the battery charger cord. Re-charge unit's battery until the "full battery" indicator's green light is illuminated indicating the battery has been fully recharged.

BUBBLE COLUMN LED LIGHT SETTINGS

The unit comes pre-set from the factory with the M3 setting of the LED lights. A Remote Control is provided if you want to change the setting. Changes should be made by the Therapist ONLY.

LED REMOTE OPERATION (Explanation of Controls)

- MODE-Press to change to next Mode
- PAUSE – Press to make LED Color stay at present; press again to move on.
- SPEED UP- Press to change speed of color change faster.
- SPEED DOWN – Press to change speed of color change slower.

LED MODES

- M1-Seven Color Flash
- M2-Leaning Color Fade
- M3- Full Color Fade
- M4-Loop
- M5-11 – Seven Color Fixation

An instruction from the manufacturer is also included with the Bubble Column LED lights Remote.

C. CLEANING / INSPECTION / MAINTENANCE

Recommended cleaning solution is a 50/50 mixture of Hydrogen Peroxide and water and/or a small amount of mild detergent. Use a soft cotton cloth in conjunction with any cleaner.

**DO NOT USE ANY CLEANING PRODUCTS CONTAINING ISOPROPYL OR ETHYL ALCOHOL.
DO NOT USE ANY AEROSOL SPRAYS AS THEY ALL CONTAIN ALCOHOL.
DO NOT USE ABRASIVE CLEANERS, AS THEY WILL SCRATCH.
DO NOT USE PAPER TOWELS AS MAY SCRATCH THE BUBBLE TUBE.**

D. MAINTENANCE

BUBBLE TUBE – Only Distilled Water should be used in the tube for operation. The water should be replaced every three (3) months with fresh Distilled Water. To drain disconnect air line from the base of the tube, remove the four mounting screws and lift out the tube to drain water. Return tube to base, secure mounting screws and reattach air line to base of tube and fill with fresh water. The Bubble Tube holds slightly less than three (3) gallons of water. Fill the Bubble Tube with the pump running to the label near the top of the tube marked “Do Not Use Alcohol to Clean” to be sure not to overfill. When adding new water 2 ounces of diluted, or a few drops of concentrated anti-algae preventer should be added to the water.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



FORM CODE: XI- 33BPW

© 2020 Southpaw Enterprises, Inc.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at
<http://southpawenterprises.com>