



Instruction Sheet

Play Tunnel 3961

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- 1 - Play Tunnel with Padded Floor
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

INSTRUCTIONS FOR USE

- Extend the Play Tunnel on the floor and turn it so that the loop fasteners are on the bottom.
- Place the padded floor inside the tunnel with the hook fasteners on the underneath side.
- Secure the padded floor to the tunnel by aligning the hook and loop fasteners and pressing them together.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Wipe with a damp cloth using soap and water or disinfectant.
- Dry before using.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Please visit us @ www.southpaw.com
© 2019 Southpaw Enterprises, Inc.