



Store these instructions in a safe place with the enclosed maintenance checklist

Instruction Sheet

Soft-Lite Linear Glider Swing 6715



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Warning: Failure to properly install a Southpaw forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the Southpaw forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- 1 - Soft-Lite Linear Glider
- 2 - Safety Snaps
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

IMPORTANT: KEEP ALL PARTS TOGETHER.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

Height Requirement: 8 feet.

Space needed: Assure a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER OR SAFETY ROTATIONAL DEVICE NEEDED

- Southpaw Height adjuster (5000) and Therapy Rope (0150) is recommended but not included for ease of adjustment to proper height.
- Southpaw Safety Rotational Device (2000) must be used if suspended from a single point.

WORKING LOAD: 300 lbs.

SUSPENSION PROCEDURES

- Attach your Soft-Lite Linear Glider to your support eye bolts with the Safety Snaps provided.
- To suspend your Soft-Lite Linear Glider from two points the suspension points must be between 3-1/2 feet to 4-1/2 feet apart. To suspend your Soft-Lite Linear Glider from four points you will need two additional Safety Snaps. The suspension points should be between 3-1/2 feet to 4-1/2 feet apart in length and 1' to 2' in width. To suspend your Linear Glider from one point: A Southpaw Safety Rotational Device must be used to prevent damage by rotation.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check Safety Snaps for wear. See the enclosed Safety Snap Instruction Sheet for inspection guidelines.
- Check the ropes for loose or missing splice locks.
- Check D-Rings and loops where ropes attach.
- Check ropes for loose or missing splice locks.* If loose or missing splice locks are found, discontinue use until repaired. * A splice lock is the metal clip near the end of the ropes.

- Check the ropes for wear at the top end where it rubs against the Safety Snaps and at the bottom end where it goes through the D-Rings. If there is excessive wear, discontinue use and call Southpaw to replace ropes.

STORAGE PROCEDURES

- Store out of direct sunlight.
- Store away from all chemicals and household cleaners.
- Store where not accessible by clients without proper supervision.
- Do not store on edge. This can cause damage to the padded edge.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Wipe vinyl with a damp cloth using soap and water or disinfectant. Dry before using.
- Do not immerse in water.

CAUTION

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047
Toll Free - (800) 228-1698
Please visit us at southpaw.com
© 2025 Southpaw Enterprises, Inc.