



# SOUTHPAW®

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

## Instruction Sheet

### Soft-Lite Platform Swing #6722 - 24" x 24"



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment. If this Instruction Sheet is misplaced, please call or write for a replacement. (See address below).

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

**Warning:** Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

#### MATERIALS LIST

- 1 - Platform Swing
- 2 - Instruction Sheets (Swing/Safety Snap)
- 1 - Maintenance Checklist

#### MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

Minimum height required is 8 ft.

Caution must be used to keep a clear distance of 6 ft. from walls and other obstacles.

#### HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

Use of the Southpaw 5000 Height Adjuster is recommended for ease of adjusting the height of your equipment.

A Southpaw 2000 Safety Rotational Device must be used to prevent damage to your equipment.

**WORKING LOAD:** 300 lbs.

#### HANGING PROCEDURE

Hang Soft-Lite Platform Swing by attaching the Safety Snap that is on the top of the Platform Swing to your height adjustment system.

#### MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check ropes for loose or missing splice locks.\* If loose or missing splice locks are found, discontinue use until repaired. \* A splice lock is the metal clip near the end of the ropes.
- Check vinyl for any tears or wear.
- Check D-rings and loops where ropes attach for wear or fraying.
- Check wood for cracks or splinters. If cracked, discontinue use. For splinters, sand until smooth.

## **CLEANING INSTRUCTIONS**

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Dust or wipe with a damp cloth.
- Vinyl pad may be cleaned with prescribed disinfectant. Dry before using.

## **STORAGE PROCEDURES**

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.
- Do not store on edge. This can cause damage to the padded edge.

## **CAUTION**

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Please visit us [southpaw.com](http://southpaw.com)

© 2024 Southpaw Enterprises, Inc.