



Instruction Sheet

Static Wrap 7011

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



WARNING! Shackles will wear with use and must be inspected on a regular basis. Discontinue use of shackle and replace or have repaired immediately when wear begins to show. Do not let wear exceed 30% of the shackle.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

1 - Static Wrap (2-XH7052, lock washer, 4-XH1104, washer, 2-XH7032, bolt, 2-XH7053, nuts, 2-XH1006, nuts)
1 - Instruction Sheet
1 - Maintenance Checklist

TOOLS NEEDED

Two 3/4" wrenches

INSTALLING THE WRAP

- Place the bracket strap (the piece without the shackle) on top of the cross beam at a safe distance from the end post. (Put the side with the shock pad against the cross beam).
- Put one flat washer on each of the hex head bolts, then put them through the holes in the hanger strap (the piece with the shackle), going through from the shackle side.
- Put the bolts up through the holes in the bracket strap. Place a flat washer, a lock washer, and nut on each of the bolts. Tighten finger tight evenly.
- Tighten the bolts by holding the head of the bolt with one 3/4" wrench while using the other wrench to tighten the nut. Alternate from one nut to the other, tightening them both evenly.

HEIGHT ADJUSTER OR SAFETY ROTATIONAL DEVICE NEEDED

A Southpaw Height Adjuster (# 5000) may be needed to adjust your equipment to the correct height for your client.

With all Southpaw suspended equipment, a Safety Rotational Device must be used to prevent damage to your equipment.

MATTING PRECAUTIONS: Always use appropriate matting.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a daily, weekly or monthly basis and recorded on the checklist enclosed.

- Try to turn the rod end on a **DAILY** basis. If it turns at all, remove and have repaired or replace immediately.
- Check to make sure that all nuts and bolts are tight on a weekly basis. If loose, discontinue use immediately until tightened.
- **WARNING!** Check for wear on the shackle. If wear exceeds 30% of the diameter of the shackle, discontinue use immediately and call Southpaw for repair or replacement.
- Use a Southpaw Safety Rotational Device (item #2000) to prevent abnormal wear and damage to equipment.

When replacing Safety Snaps or Rotational Devices due to normal wear, use only Southpaw replacement parts.

CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Wipe with a dry cloth.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

CAUTION

- ♦ **Check shackle for wear. Discontinue use of shackle and replace immediately when wear begins to show. Do not let wear exceed 30% on the shackle. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047
Toll Free - (800) 228-1698
Please visit us at southpaw.com
© 2025 Southpaw Enterprises, Inc.