



Instruction Sheet

ITN Platform Swing 7260

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- | | |
|------------------------|-----------------------------------|
| 1 - ITN Platform Swing | 1 - Carrying Case |
| 2 - Spreader Bars | 1 - Instruction Sheet |
| 4 - Lynch Pin Clips | 1 - Safety Snap Instruction Sheet |
| 1 - Safety Snap | 1 - Maintenance Checklist |

WORKING LOAD: 300 lbs.

INSTRUCTIONS FOR USE:

- The ITN Platform Swing is designed for use with the Southpaw ITN Frame. Attach the Safety Snap to the swivel eye bolt at the top of the frame. Slide the welded ring on the ITN Platform Swing ropes up to the bottom of the Safety Snap.
- To remove the wood spreader bars, swing the rings on the lynch pins open as shown in figure 1. Pull the pins out of the bar and remove the bar from the ropes..
- To put the bars back on the swing, insert one end of one of the spreader bar onto one of the ropes *just above the vinyl tube*. Open the ring on a lynch pin clip and put the pin through the 1/4" hole near the end of the bar making sure that the rope is retained by the pin. (See fig. 1). Swing the ring around the end of the bar and it will snap closed trapping the rope in the slot. (See fig. 2). Repeat this procedure with the other end of the bar to the opposite rope on the same side of the swing. Attach the other bar to the other two ropes the same way.
- When storing the swing, slide the welded ring down over the ropes to keep the ropes bundled together.

MAINTENANCE CHECKLIST:

All of the above wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the eye bolt/safety snap connection point for wear. (See Safety Snap Instruction Sheet).
- Check ropes for loose or missing splice locks*. If loose or missing splice locks are found, discontinue use until repaired.
- Check edge padding to see that it is securely attached.
- Check wood for cracks or splinters. If cracks or splinters are found, discontinue use until repaired.

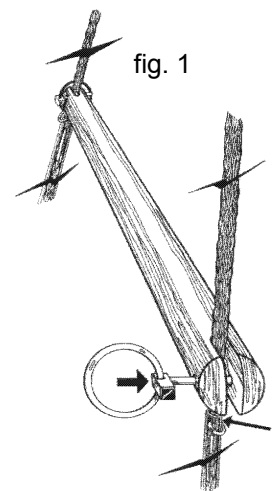


fig. 1

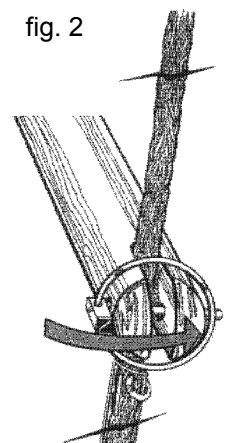


fig. 2

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Dust or wipe with a damp cloth.
- Vacuum carpet. Carpet may be cleaned with foam carpet cleaner. Carpet is not machine washable.

CAUTION

- ◆ **Check eye bolt for wear and movement.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Please visit us @ www.southpaw.com
© 2019 Southpaw Enterprises, Inc.