



Store these instructions in a safe place or with the enclosed maintenance checklist

## Instruction Sheet Kidlite Barrel 1050



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST

- 1 - Kidlite Barrel
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

### SPECIAL NOTES

- Kidlite Barrel is to be used with only one client at a time.
- Do not allow clients to stand on barrel.
- Do not use with any ramp or platform.
- Use caution so clients fingers do not get pinched where the barrel meets the floor.

**WORKING LOAD:** 250 lbs.

### MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the vinyl covering on the padded edges and on the inside. If the vinyl covering has rips or tears, discontinue use until repaired. CAUTION: Do not allow clients to pick foam from inside and ingest it.
- Check for protruding staples. If found, remove the staple. To check for loose staples, gently run your fingers around the inside edge of the padded rim to feel for protruding staples. Check carpet for protruding staples by patting the the entire surface with open hands. Most of the staples are along the edges, but a few are scattered throughout the carpet to secure it to the barrel. Protruding staples can be removed by pulling carefully with a pair of pliers.
- Check carpet to be sure that the edges are tightly stapled down. If carpet is loose, call Southpaw for repairs.

### STORAGE PROCEDURES

- Store out of direct sunlight.
- It is best to store the Kidlite Barrel in an upright position.
- Do not store near chemicals or household cleaners.
- Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Vinyl surfaces may be cleaned with a damp cloth using soap and water or disinfectant.
- Dry before using.
- Vacuum carpet often.
- Carpet may be cleaned with dry carpet cleaner.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Come see us on the World Wide Web at <http://southpawenterprises.com>



southpaw enterprises, inc.®

Store these instructions in a safe place or with the enclosed maintenance checklist

# Instruction Sheet

## Soft-Lite Barrel 1060



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please find this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST

- 1 - Soft-Lite Barrel
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

### SPECIAL NOTES

- Use caution if using Soft-Lite Barrel with more than one client at a time.
- Do not allow clients to stand on barrel.
- Do not use with any ramp or platform.
- Although the barrel is made of foam, use caution that fingers do not get pinched where barrel meets the floor.
- The foam used in the Soft-Lite Barrel is constructed of very sturdy foam, however if it is flattened or deformed severely, it may not recover it's original shape.

**WORKING LOAD:** 225 lbs.

### MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the vinyl cover for loose stitches, rips, or tears. If loose stitches, rips, or tears are found, discontinue use until repaired.  
CAUTION: Do not allow clients to pick foam from inside and ingest it.

### STORAGE PROCEDURES

- Store out of direct sunlight.
- It is best to store the barrel in an upright position.
- Do not store near chemicals or household cleaners.
- Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Wipe with a damp cloth using soap and water or disinfectant.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>