



Store these instructions in a safe place or with the enclosed maintenance checklist

Instruction Sheet Twirler 1149



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST

- 1 - Twirler (rope and handle)
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

HEIGHT REQUIREMENTS: 8 ft. A clear distance of 6 ft. must be kept from walls and other obstacles.

WORKING LOAD: 300 lbs.

HANGING INSTRUCTIONS

Suspend from your suspension point with a Safety Snap (0100) through the metal lined loop at the top of the white rope.

- **IMPORTANT:** For extra safety margin, make sure to leave your Safety Rotational Device (2000) in place.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

Contact Southpaw for repair or replacement of worn or frayed parts immediately.

- Check for wear on metal lining of rope at top and under the eye bolt cover.
 - If rotation stops or makes noise, STOP using immediately. Return to Southpaw for repair.
 - Check the ropes for loose or missing splice locks*.
 - Foam grips on handle must be tight.
- * A splice lock is the metal rope staple near both ends of the rope.

SPECIAL NOTE: Use southpaw height Adjuster (5000) for ease in adjusting up and down.


STORAGE PROCEDURES

- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Clean with a damp cloth using soap and water or disinfectant.
- Dry before using.
- Do not immerse in water.

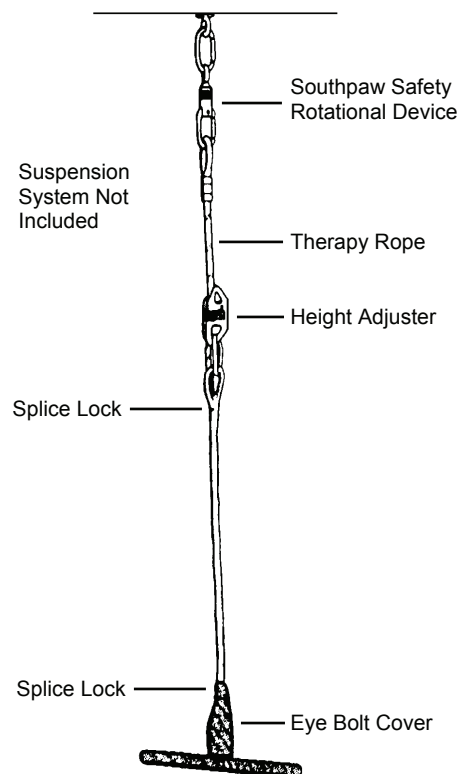

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>