



southpaw enterprises, inc.®

Instruction Sheet

Giddie Up Glider 120010

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- 1 - Giddie Up Glider w/ safety snaps
- 2 - Instruction Sheets
- 1 - Maintenance Checklist

INSTRUCTIONS FOR USE:

- To use with the Southpaw Home Therapy System, attach the front and rear lines on the left side to a Southpaw Safety Snap. Attach the front and rear lines on the right side to another safety snap. Attach the snaps to the web loops on the bar.
- To use with a two point suspension system with suspension points between 24" to 48" apart, attach the two rear lines to a Southpaw Safety Snap. Attach the two front lines to another safety snap and hang the glider. In this configuration the front will be higher than the rear. You can use another snap or a Southpaw Height Adjuster (# 5000) on the front to get the glider to hang level.

CAUTION:

- The pivot point at the front of the glider is a potential pinch point. Keep fingers and legs away.
- The foot pegs are not meant to support the full weight of rider. Do not stand on pegs.

WORKING LOAD: 200 lbs.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check ropes for loose or missing splice locks.* If loose or missing splice locks are found, discontinue use until repaired.
- Check wood for cracks or splinters. If cracked, discontinue use. For splinters, sand smooth and refinish with clear lacquer.
- Check to be sure that acorn nut at pivot point is tight.
- Check seat padding to see that it is securely attached.

* A splice lock is the metal clip near the end of the ropes.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Dust or wipe with a damp cloth.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>