



southpaw enterprises, inc.®

Instruction Sheet

Infant Swing 120020

Not-So-Infant Swing 120025

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST:

- | | |
|--------------------------------|-----------------------------------|
| 1 - Infant Swing w/Safety Snap | 1 - Safety Snap Instruction Sheet |
| 1 - Instruction Sheet | 1 - Maintenance Checklist |

INSTRUCTIONS FOR USE:

To use the Southpaw Infant Swing, attach the Southpaw Safety Snap (included) to your suspension point. A Southpaw Height Adjuster System (# 410001) may be needed to adjust the swing to the proper height for your client. A Southpaw Safety Rotational Device (# 2000) must be used for any rotational activity.

- *CHECK TO BE SURE THAT SEAT BUCKLES UNDER WOOD FRAME ARE FASTENED SECURELY.*
- *POSITION CLIENT IN SWING FACING THE STRAIGHT EDGE SIDE OF THE SWING.*
- *DO NOT LEAVE CLIENT UNATTENDED IN SWING.*

SPACE REQUIREMENTS: Use caution to keep a minimum of 6 foot clear distance from walls and other obstacles.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the Safety Snap for wear. (See Safety Snap Instruction Sheet).
 - Check ropes for fraying and loose or missing splice locks.* If loose or missing splice locks are found, discontinue use until repaired.
 - Check wood for cracks or splinters. If cracked, discontinue use. For splinters, sand until smooth.
 - Check edge padding to see that it is securely attached.
- * A splice lock is the metal clip near the end of the ropes.
- Check seams for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.

MATTING PRECAUTIONS:

- Always use appropriate matting.
- Crash mats and wall mats must be used with all hanging equipment.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Machine wash warm water, mild detergent. Tumble dry, low heat.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>