



southpaw enterprises, inc.®

# Instruction Sheet

## FITBALL WEDGE - 1332

## FITBALL WEDGE JR - 1333

Please store this instruction sheet in a safe place or access it on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

**IMPORTANT: Keep all parts together.**

**INFLATION:** To adjust firmness, remove the plug located on the wide end of the wedge. Inflate or deflate to desired firmness and replace the plug. Use only a hand pump. A pump suitable for therapy balls can be used. DO NOT USE HIGH PRESSURE AIR.

**CAUTION: Please keep the plug away from small children when ball is not inflated. The plug is small and could be swallowed.**

### STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Clean with a damp cloth using soap and water or disinfectant.
- Dry before using.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Come see us on the World Wide Web at <http://southpawenterprises.com>