



southpaw enterprises, inc.®

Instruction Sheet

Store these instructions in a safe place or with the enclosed maintenance checklist

Tactile Massage Roll - 2 Pack 260010

Tactile Massage Ring - 2 Pack 260011



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- 2 - Tactile Massage Rolls / Rings
- 1 - Instruction Sheet

NOTE: Inflate to desired firmness using the inflating stem provided. When inflated, remove the inflating stem and insert the plug. Use only a hand pump. A pump suitable for therapy balls such as the Double Quick Pump or the Inflator / Deflator offered by Southpaw can be used. **DO NOT USE HIGH PRESSURE AIR.**

CAUTION: THE PLUG IS SMALL ENOUGH THAT IT COULD BE SWALLOWED BY SMALL CHILDREN.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Clean with a damp cloth using soap and water or disinfectant.
- Dry before using.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401-1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>