



southpaw enterprises, inc.®

Store these instructions in a safe place or you may access them on our website.

# Instruction Sheet

## Sensory Rollers 2861



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST:

- 1 - Single Hand Roller with removable ball.
- 1 - Double Head Roller with removable ball.
- 1 - Four Roller Wand non-removable balls

1 - Instruction Sheet

**Note:** The Sensory Rollers are made from natural rubber. Removable balls may be used independently.

**IMPORTANT:** Keep all parts together.

### THINGS TO CHECK PERIODICALLY:

- Make sure that the rollers are free of dirt, hair, or other objects. Clean out if necessary.

### STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Single and double init is top shelf dishwasher safe.
- Do not immerse four roller wand in water.
- Clean with a damp cloth using soap and water or disinfectant. Dry immediately.
- Always dry before using.
- To prevent rubber rollers from drying out, treat them with a rubber protectant.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>