



**southpaw enterprises, inc.®**

# Instruction Sheet: Stereo Rover 303100

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

## MATERIALS LIST:

Stereo Rover (completely assembled)  
2 Keys for Lock in Cabinet

## INSTRUCTIONS FOR USE:

Stereo Rover was intended for use with Southpaw Vibro Acoustic products (Rocker, Wedge, Ball Pool, Padded Corner, and Waterbed). Any two of these items can be connected to the Stereo Rover at one time. Connections are best used with the cords supplied with vibro product. Cord has one RCA jack at each end. Plug one end into the vibro product and the other into the rear of the Stereo rover. If standard speaker wire is used match the red terminals and the black terminals. Refer to the Stereo instruction manual for operation of Stereo. For best results, when using vibro products and the external speakers built into the rover press the Audio button on the remote once to get to the FADE option on the Stereo. Press the Left Arrow button until the FADE REAR plus 11 appears. Stereo can be used only with CD's or an MP3 player.

## MAINTENANCE CHECKLIST

✓All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check for frayed wires
- Check the wood surface and edges for splinters. If found, discontinue use until repaired. You may repair it by sanding the area smooth with a medium grit sandpaper, and applying two coats of clear lacquer to the repaired area.

## STORAGE PROCEDURES

- Store where not accessible by clients without proper supervision.

## CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean with a damp rag and a mild surface cleaning solution
- Let air dry or wipe dry with a cloth.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>



**southpaw enterprises, inc.®**

# Instruction Sheet: Little Radio Rover 303199

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

## MATERIALS LIST:

Radio Rover (completely assembled)  
2 Keys for Lock in Cabinet

## INSTRUCTIONS FOR USE:

Radio Rover was intended for use with Southpaw Vibro Acoustic products (Rocker, Wedge, Ball Pool, Padded Corner, and Waterbed). Any two of these items can be connected to the Radio Rover at one time. Connections are best used with the cords supplied with vibro product. Cord has one RCA jack at each end. Plug one end into the vibro product and the other into the rear of the radio rover. If standard speaker wire is used match the red terminals and the black terminals. Refer to the radio instruction manual for operation of radio. For best results, when using vibro products and the external speakers built into the rover press the Audio button on the remote once to get to the FADE option on the radio. Press the Left Arrow button until the FADE REAR plus 11 appears. Radio can be used only with CD's or an MP3 player.

## MAINTENANCE CHECKLIST

✓All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check for frayed wires
- Check the wood surface and edges for splinters. If found, discontinue use until repaired. You may repair it by sanding the area smooth with a medium grit sandpaper, and applying two coats of clear lacquer to the repaired area.

## STORAGE PROCEDURES

- Store where not accessible by clients without proper supervision.

## CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean with a damp rag and a mild surface cleaning solution
- Let air dry or wipe dry with a cloth.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>