



**southpaw enterprises, inc.®**

# Instruction Sheet: Vibro-Acoustic Recliner #303260

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

## MATERIALS LIST:

- 1 - Vibro-Acoustic Recliner
- 1 - 20' Speaker Wire

## INSTALLATION INSTRUCTIONS :

The Recliner comes delivered completely assembled. In order to make the Recliner vibrate you must connect it to a Southpaw Radio Rover, Mini Rover, Wall or Corner Radio. Use with any other stereo equipment will void the warranty.

The speaker wire comes with RCA jacks at each end. Locate the speaker connection on the side of the chair and plug one end of the wire into the RCA socket. Connect the other end of the wire to the RCA socket on a Radio Rover.

## MAINTENANCE:

Recommended cleaning solution is 50/50 solution of Hydrogen Peroxide to water including a teaspoon of dish detergent per gallon. A soft cotton cloth should also be used.

Care should be taken to remove all sharp objects from pockets, as well as all metal braces or braces containing metal snaps or connectors, prior to sitting on the Recliner.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw  
Enterprises Inc.®**

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>