



Instruction Sheet Static Eye Bolt Wrap 7011

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



WARNING! Shackles will wear with use and must be inspected on a regular basis. Discontinue use of shackle and replace or have repaired immediately when wear begins to show. Do not let wear exceed 30% of the shackle.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

1 - Static Eye Bolt Wrap
1 - Instruction Sheet
1 - Maintenance Checklist

TOOLS NEEDED

Two 3/4" wrenches

INSTALLING THE EYE BOLT WRAP

- Place the bracket strap (the piece without the shackle) on top of the cross beam at a safe distance from the end post. (Put the side with the shock pad against the cross beam).
- Put one flat washer on each of the hex head bolts, then put them through the holes in the hanger strap (the piece with the shackle), going through from the shackle side.
- Put the bolts up through the holes in the bracket strap. Place a flat washer, a lock washer, and a non-locking nut on each of the bolts. Tighten finger tight evenly.
- Tighten the bolts by holding the head of the bolt with one 3/4" wrench while using the other wrench to tighten the nut. Alternate from one nut to the other, tightening them both evenly.
- When both bolts are tightened securely, put a self-locking nut on each bolt and tighten it down firmly against the non-locking nut.

HEIGHT ADJUSTER OR SAFETY ROTATIONAL DEVICE NEEDED

A Southpaw Height Adjuster (# 5000) may be needed to adjust your equipment to the correct height for your client.

With all Southpaw suspended equipment, a Safety Rotational Device must be used to prevent damage to your equipment.

MATTING PRECAUTIONS: Always use appropriate matting.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a daily, weekly or monthly basis and recorded on the checklist enclosed.

- Try to turn the eye bolt on a **DAILY** basis. If it turns at all, remove and have repaired or replace immediately.
- Check to make sure that all nuts and bolts are tight on a weekly basis. If loose, discontinue use immediately until tightened.
- **WARNING!** Check for eye bolt wear. If wear exceeds 30% of the diameter of the eye bolt, discontinue use immediately and call Southpaw for repair or replacement.
- Use a Southpaw Safety Rotational Device (item #2000) to prevent abnormal wear and damage to equipment.

When replacing Safety Snaps or Rotational Devices due to normal wear, use only Southpaw replacement parts.

CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Wipe with a dry cloth.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

CAUTION

- ◆ **Check eye bolt for wear and movement. Discontinue use of eyebolt and replace immediately when wear begins to show. Do not let wear exceed 30% of eye bolt. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.®**

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>



Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet Swivel Wrap 7012



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

1 - Swivel Wrap
1 - Instruction Sheet
1 - Maintenance Checklist

TOOLS NEEDED

Two 3/4" wrenches

INSTALLING THE EYE BOLT WRAP

- Place the bracket strap (the piece without the shackle) on top of the cross beam at a safe distance from the end post. (Put the side with the shock pad against the cross beam).
- Put one flat washer on each of the hex head bolts, then put them through the holes in the hanger strap (the piece with the eye bolt), going through from the shackle side.
- Put the bolts up through the holes in the bracket strap. Place a flat washer, a lock washer, and a non-locking nut on each of the bolts. Tighten finger tight evenly.
- Tighten the bolts by holding the head of the bolt with one 3/4" wrench while using the other wrench to tighten the Nylon locking nut. Alternate from one nut to the other, tightening them both evenly.
- When both bolts are tightened securely, put a self-locking nut on each bolt and tighten it down firmly against the Nylon locking nut.

HEIGHT ADJUSTER OR SAFETY ROTATIONAL DEVICE NEEDED

A Southpaw Height Adjuster (# 5000) may be needed to adjust your equipment to the correct height for your client.

With all Southpaw suspended equipment, a Safety Rotational Device must be used to prevent damage to your equipment.

MATTING PRECAUTIONS: Always use appropriate matting.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check to make sure that all nuts and bolts are tight. If loose, discontinue use immediately until tightened.
- Check to make sure that the eye bolt turns smoothly. If the eye bolt makes a noise while spinning, lubricate with a grease gun at the grease fitting above the eye bolt. Add grease slowly until grease starts to appear where the eye bolt turns in the bracket.

- **WARNING!** Check for eye bolt wear. If wear exists, it should not exceed 30% of the diameter of the shackle. If it does, discontinue use immediately and call Southpaw for repair or replacement.

When replacing Safety Snaps or Rotational Devices due to normal wear, use only Southpaw replacement parts.

CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Wipe with a dry cloth.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Do not let wear exceed 30% of eye bolt. If the eye bolt moves excessively, discontinue use until all hardware is checked and tightened. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>



southpaw enterprises, inc.®

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet

Replacement Eyebolt Kit 7015



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises

MATERIALS LIST:

- 1 - Eyebolt 1/2-13 x 6"
- 1 - 1/2" Flat Washer
- 1 - 1/2-13 Lock Nut

TO REMOVE WORN EYEBOLT:

Insert a large screwdriver through the "eye" of the eyebolt to keep it from turning while you loosen the nut using a 3/4" wrench. Remove the eyebolt and discard the used parts. Do not reuse the old lock nut or washer.

TO INSTALL NEW EYEBOLT:

Insert the new eyebolt into the beam from the bottom to the top. Put the 1/2" washer on the bolt. Start the 1/2" lock nut on the bolt. Insert the screwdriver through the "eye" of the eyebolt to keep it from turning while you tighten the nut. When the nut is almost tight, turn the screwdriver to properly align the eyebolt and tighten the nut securely.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Client should always wear a safety helmet with all suspended equipment.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.®**

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698